




Refer to Forest General Information Operator's Guide for Glossary of Symbols, Terms, EMC/Electrical Safety Declaration and User Guidance.

Indications for Use/Intended Use The dental stool is intended to position the dental practitioner for access and visibility to the oral cavity during diagnostic and therapeutic treatment in a clinical environment. There are no contraindications for this product.

⚠ CAUTION: Federal law restricts this device to sale by or on the order of a dental practitioner licensed by the law of the State in which he/she practices to use or order the use of the device.


⚠ WARNING: Refer to Forest Operatory Equipment Asepsis Operator's Guide for surface and barrier protection procedures.

6156C Doctor's Economy Stool, 6157C Assistant's Economy Stool and 6160C Assistant's Deluxe Stool

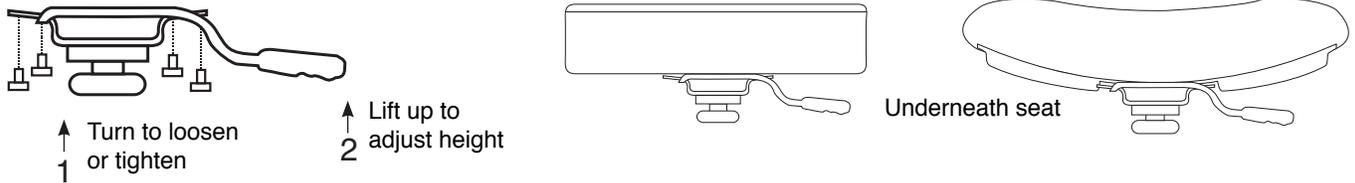

1. Rotate backrest height adjustment knob counter clockwise to loosen backrest. Once desired position is reached, rotate backrest height adjustment knob clockwise to lock backrest in place.
2. Lift up on right lever to adjust seat height. Release lever to lock in place.

6159C Doctor's Deluxe Stool and 6161C Assistant's Deluxe Stool

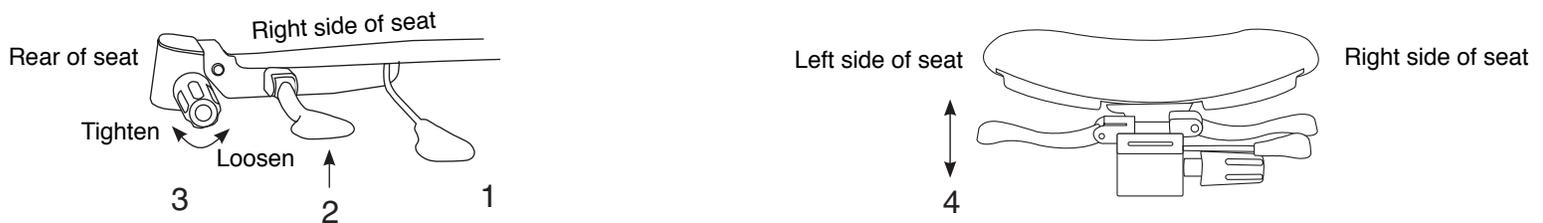

1. Lift up on front right lever to adjust seat height.
2. Lift up on rear right lever to adjust seat tilt.
3. Rotate backrest height adjustment knob counter clockwise to loosen backrest. Once desired height is reached, rotate backrest height adjustment knob clockwise to lock backrest in place.
4. Lift up on left lever to adjust backrest tilt. Push down on left lever until lever locks into place to actuate the free floating backrest.



Forest Dental Equipment



DTE Oregon, Inc.
Address 6200 ne cherry drive hillsboro oregon 97124 • USA

Phone 800 . 423 . 3555 • 503 . 640 . 3012 | Fax 503 . 693 . 9715 | Digital sales@forestdental.com • forestdental.com



4.13.19 • 0097-524 • Rev. B • 2 of 2