

Model Family	Model Numbers	User Guide Name	Version	Notes	Oil Type	Oil Service	Oil Filter	Air Filters
Standard 550 Standard 1200 Standard 1400 Standard 2800	955.####.## 912.####.## 914.####.## 928.####.##	Standard User Guide	Version 93a	For all Standard (Original Chassis), Non-Recirculating	Mobile Delvac 15w40	Drain used oil regularly. Add fresh oil as necessary to maintain site glass ½ to ¾ full.	Clean or Replace	Clean or Replace
Recirculating 1400	914.####.##R	Recirculating Oil Systems User Guide	Version 95a	For all Standard (Original Chassis), Recirculating (factory configured) and Recirculating (field converted)		Change Oil 6 quarts Site glass ½ to ¾ full		
Bulldog	855.####.##	Bulldog User Guide	Version 98a	For all Bulldog (not Bulldog QT) models	Mobil 1 15w50	Change Oil Less than 5 quarts Site glass ½ to ¾ full	Clean or Replace	Replace
Bulldog QT	809.0800.## 809.1050.## 809.1450.##	Bulldog QT User Guide	Version 2000A	Bulldog QT1 & QT2 S/N QT0012175 and prior		Change Oil 3 quarts Site glass ½ to ¾ full		
		Bulldog QT User Guide	Version 2001A	Bulldog QT1 & QT2 S/N QT0012176 and later For all Bulldog QT3		Change Oil 3 quarts Site glass ½ to ¾ full		
Bulldog QT with S1 Electrols	809E.####.#	Bulldog QT User Guide For Bulldog QT with S1 Electrols	Version 01-Jul-06	For all Bulldog QT with S1 Electrols	Change Oil 3 quarts Site glass ½ to ¾ full			
Bison	818.####.## 826.####.##	Bison User Guide	Version 2000A	For Bison S/N BN0012112 and prior	Mobil 1 15w50	Change Oil 6 quarts Site glass ½ to ¾ full	Clean or Replace	Replace
		Bison User Guide	Version 2001A	For Bison S/N BN0012113 and later		Change Oil 5 quarts Site glass ½ to ¾ full.		
Bison with S1 Electrols	826E.####.#	Bison User Guide For Bison with S1 Electrols	Version 2002A	For Bison 7 & 9 S/N BN0309139 and prior		Change Oil 5 quarts Site glass ½ to ¾ full		
Bison with S1 Electrols	818E.####.# 826E.####.#	Bison User Guide For Bison with S1 Electrols	Version 01-Jul-06	For all Bison 3 & 5 For Bison 7 & 9 S/N BN0301140 and later		Change Oil 5 quarts Site glass ½ to ¾ full		



RAMVAC  
3100 First Avenue  
Spearfish SD 57783

(800) 5-RAMVAC  
(800) 572-6822  
(605) 642-4614

Fax (605) 642-3776  
ramvac@ramvac.com

Q:\SOF\SOF\_7.2.3-02-140.doc  
Effective Date 24-Jul-06  
Page 1 of 1 Pages