

How to replace dripper cartridge in all models.

1. Secure disconnect.
2. Loosen oil filter to stop oil flow to dripper.
3. Disconnect oil line (if present) from brass nut.
4. Remove brass nut from dripper tower.
5. Remove faulty dripper cartridge.

Note: If dripper cartridge o-ring is not removed from dripper tower oil will flow through the assembly without proper control.

6. Verify the dripper cartridge O-ring is present on the bottom end of faulty dripper cartridge. If not present on dripper cartridge it must be removed from inside dripper tower.
7. Verify new O-ring is properly seated in groove located in bottom of replacement dripper cartridge.
8. Install dripper cartridge with the O-ring end facing down.
9. Fill the dripper tower with proper oil type.
10. Install brass nut in dripper tower.
11. Connect oil line (if present) to brass nut.
12. Hand tighten oil filter.
13. Verify proper drip rate (allow unit to run 30 minutes).

