

SLUGBUSTER™ DENTAL VACUUM LINE CLEANER

NOTE: SlugBuster is a non-foaming, non-chlorinated dental vacuum line cleaner that is compatible with ALL dental vacuum systems and amalgam separators (8 pH).

NOTICE

Ensure that vacuum system is turned **OFF** when indicated.

CLEAN VACUUM SYSTEM LINES

Daily

1. Turn **OFF** vacuum system.
2. Clean or replace solids separator screen in operatory.
3. Turn vacuum system back **ON**.
4. Rinse vacuum lines by drawing one quart of **HOT** water through each high-volume evacuation (HVE) line and a few ounces of **HOT** water through each saliva ejector (SE) line.
5. Draw a few ounces of SlugBuster through each HVE line and a few ounces through each SE line (SlugBuster pre-mixed 1 ounce per 2 gallons of water).



During Surgery

Every few minutes, draw a few ounces of water through aspirator(s) used.

Following Surgery

Draw several ounces of SlugBuster through aspirator(s) used.

HELPFUL HINTS

- Prophy agents, pumice and fluoride gels tend to accumulate in vacuum lines.
- Flushing lines with ample quantities of HOT water is the most effective and economical way to assist in their removal.
- Increase quantity of SlugBuster used to remove heavy deposits from solid separators and hoses.
- Finish each operatory by opening and closing HVE several times to thoroughly clean risers to overhead lines.
- Rinsing and using vacuum line cleaners will not eliminate need to empty solids separators in the operatory.
- It is not necessary to rinse vacuum lines that have been inactive.

NOTICE

SlugBuster is available in powder and liquid forms, and can be ordered through any authorized Ramvac® dealer.

RAMVAC PART NO.

CONTENTS

900143	Powder Starter Pack: Mixing jug, two 40-ounce containers of SlugBuster
900146	Powder Six Pack: Six 40-ounce containers of SlugBuster (makes 480 gallons of solution)
900241	Liquid Single Gallon: One gallon of SlugBuster concentrate and hand pump dispenser
900243	Liquid Starter Pack: Mixing jug, one gallon of SlugBuster concentrate and hand pump dispenser
900246	Liquid Four Pack: Four single gallons of SlugBuster concentrate and four hand pump dispensers