

### **SLUGBUSTER™**, Dental Vacuum Line Cleaner

**NOTE:** *SlugBuster™ is a non-foaming, non-chlorinated dental vacuum line cleaner that is compatible with ALL dental vacuum systems and Amalgam Separators (8 pH)*

#### Cleaning Instructions

### NOTICE

Ensure that vacuum system is turned off when indicated.

#### Daily

1. Turn **OFF** vacuum system.
2. Clean or replace solids separator screen in operatory.
3. Turn vacuum system back **ON**.
4. Rinse vacuum lines by drawing one quart of **HOT** water through each high volume evacuation line (HVE) and a few ounces of **HOT** water through each saliva ejector line (SE).
5. Next draw a few ounces of SlugBuster through each HVE line and a few ounces through each SE line. (SlugBuster pre-mixed 1 oz. per 2 gal. water)

#### During Surgery


1. Every few minutes, draw a few ounces of water through aspirator(s) used.

#### Following Surgery

1. Draw several ounces of SlugBuster through aspirator(s) used.

#### Helpful Hints

- Prophy agents, pumice and fluoride gels tend to accumulate in vacuum lines.
- Flushing lines with ample quantities of **HOT** water is most effective and economical way to assist in their removal.
- Increase quantity of SlugBuster used to remove heavy deposits from solid separators and hoses.
- Finish each operatory by opening and closing HVE several times to thoroughly clean risers to overhead lines.
- Rinsing and using vacuum line cleaners will not eliminate need to empty solids separators in the operatory.
- It is not necessary to rinse vacuum lines that have been inactive.

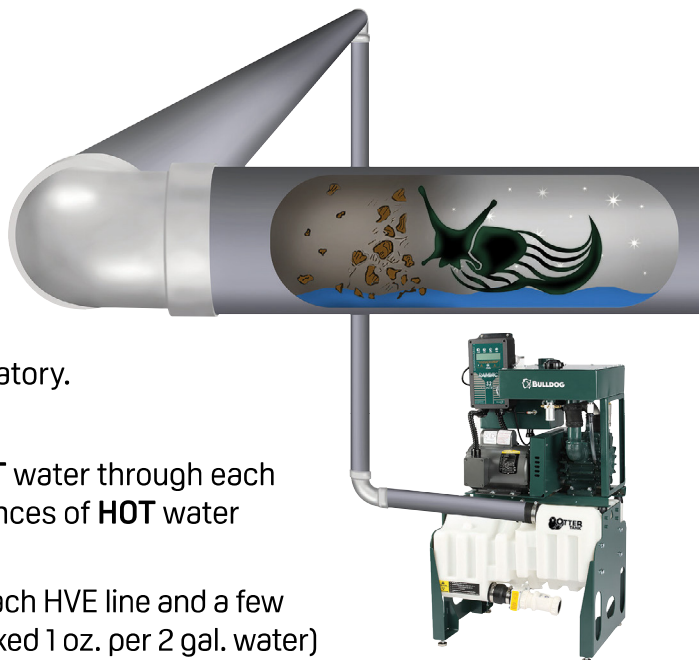

### NOTICE

Reorder your SlugBuster from your Authorized RAMVAC Dealer.