

Sopro 617 Camera

MAINTENANCE

For each new patient, it is essential to use the dental barriers provided with the camera or provided as a Complement

Before first using it, it is imperative to follow the complete disinfecting procedure

Any intra-oral camera returning from service or maintenance should follow the complete disinfecting procedure before being used.

***WARNING: Do not use products containing the following - :
Deterioration risks of the plastic parts***

- *Ammoniac, trichloroethylene*
- *Dichloroethylene*
- *Ammonium hydrochlorid*
- *Chlorinated and aromatic hydrocarbon*
- *Ethylene dichloride*
- *Methylene chloride*
- *Ketones.*

HANDPIECE AND CONNECTION BOX MAINTENANCE:

DESCRIPTION	RECOMMENDATIONS	USE INSTRUCTIONS AND PRECAUTIONS	
		✓	✗
Disinfecting.	<ul style="list-style-type: none"> • Surface cleaning and disinfecting wipes of Wipes Septol™ type - Pierre Rolland. 	<ul style="list-style-type: none"> ✓Take the wipe, wring it, and then scrub the equipment until obtaining visible cleanliness. ✓Allow to dry in the open air. ✓Carefully close the packaging box. 	<ul style="list-style-type: none"> ✗ Do not scrub ✗ Do not rinse. ✗ Do not immerse in a disinfecting liquid.