

SLUGBUSTER™ DENTAL VACUUM LINE CLEANER

NOTE: SlugBuster is a non-foaming, non-chlorinated dental vacuum line cleaner that is compatible with ALL dental vacuum systems and amalgam separators (8 pH).



NOTICE

Ensure that vacuum system is turned **OFF** when indicated.

CLEAN VACUUM SYSTEM LINES

Daily

1. Turn **OFF** vacuum system.
2. Clean or replace solids separator screen in operatory.
3. Turn vacuum system back **ON**.
4. Rinse vacuum lines by drawing one quart of **HOT** water through each high-volume evacuation (HVE) line and a few ounces of **HOT** water through each saliva ejector (SE) line.
5. Using a graduated measuring cup, draw 1 cup (8 ounces) of SlugBuster Liquid or SlugBuster pre-mixed (1 ounce per 2 gallons of water) through each HVE and SE line.



During Surgery

Every few minutes, draw a few ounces of water through aspirator(s) used.

Following Surgery

Draw several ounces of SlugBuster through aspirator(s) used.

HELPFUL HINTS

- Prophy agents, pumice and fluoride gels tend to accumulate in vacuum lines.
- Flushing lines with ample quantities of HOT water is the most effective and economical way to assist in their removal.
- Increase quantity of SlugBuster used to remove heavy deposits from solid separators and hoses.
- Finish each operatory by opening and closing HVE several times to thoroughly clean risers to overhead lines.
- Rinsing and using vacuum line cleaners will not eliminate need to empty solids separators in the operatory.
- It is not necessary to rinse vacuum lines that have been inactive.

NOTICE

SlugBuster is available in powder and liquid forms, and can be ordered through any authorized Ramvac dealer.

RAMVAC PART NO.	CONTENTS
900143	Powder Starter Pack: Mixing jug, two 40-ounce containers of SlugBuster
900146	Powder Six Pack: Six 40-ounce containers of SlugBuster (makes 480 gallons of solution)
900241	Liquid Single Gallon: One gallon of SlugBuster concentrate and hand pump dispenser
900243	Liquid Starter Pack: Mixing jug, one gallon of SlugBuster concentrate and hand pump dispenser
900246	Liquid Four Pack: Four single gallons of SlugBuster concentrate and four hand pump dispensers