866.DTE.INFO (383.4636)  
www.DentalEZ.com

62491 F Fecha de Publicación 2018/01/30

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866.DTE.INFO (383.4636)  
www.DentalEZ.com

StarDental® Instruments

1816 Colonial Village Lane  
Lancaster, PA 17601 U.S.A.  
Printed in the USA  
2018 DentalEZ, Inc.

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**Limited Warranty**

**StarDental® Concentrix Manual Chuck Turbine and O-Rings Replacement Kit**, is warranted against defects in material and workmanship for 3 months from date of purchase.

THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTIES OF MERCHANTABILITY AND FITNESS FOR USE.

StarDental® Instruments

StarDental®  
1816 Colonial Village Lane  
Lancaster, PA 17601 U.S.A.  
Printed in the USA  
© 2018 DentalEZ, Inc.

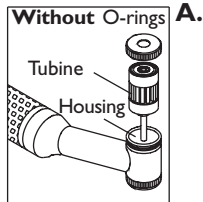
**WARRANTY REPAIR/REPLACEMENT POLICY:** Repairs/Replacements made under the new item warranty are warranted only for the remainder of the original new warranty. Repairs/Replacements made after the original new item warranty period carry the warranty above.

# Concentrix®

## Turbine & O-Rings Replacement

### INSTRUCTIONS:

**Note:** Handpieces bearing serial numbers A through I do not require O-rings when changing the turbine. Follow procedure in Section **A**.

- A.**
1. Push cartridge out back of handpiece.
  2. Clean housing with a cotton swab soaked in isopropyl alcohol.
  3. Install new turbine.
  4. Thread end cap on.




### **B.** NEW With O-rings

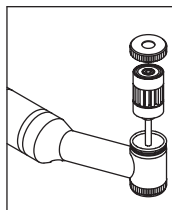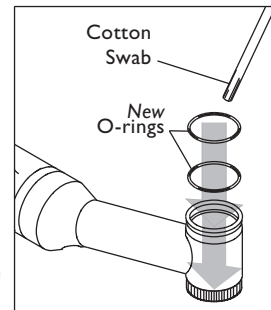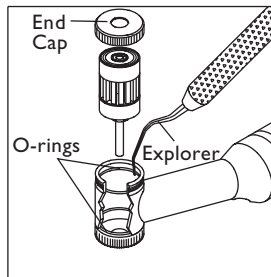1. Remove end cap by turning counterclockwise.
2. Push cartridge out back of handpiece.
3. Remove the O-rings using an instrument such as an explorer.
4. Clean housing with a cotton swab soaked in isopropyl alcohol.
5. Use blunt end of cotton swab to install new O-rings.
6. Apply *DentaLube® II*, **StarDental® Part #262539**, to inside of O-rings with soft end of cotton swab before installing turbine.

**CAUTION:** Take care to prevent damage to housing and O-rings when installing O-rings.

7. Install new turbine.
8. Thread end cap on.

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