

StarDental® Instruments

# DentaLube for Titan® Blis-sonic™ Scalers



Follow instruction manual for recommended lubrication procedure.

**CAUTION: Do not use in LubeFree Titan Motors or LubeFree Angles.**

## Aerosol Lubricant and Conditioner

DentaLube for Titan Blis-sonic Scalers

Contains a petroleum lubricant oil (CAS 64742-54-70), Heptane (CAS 142-82-5), and CO<sub>2</sub> (CAS 124-38-9).

READ BACK PANEL CAREFULLY.

To reorder use:  
StarDental® Part #261818



CFC FREE

CONTENTS: 170 GRAMS. (6 OZ. AVDP)  
8.0 FL. OZ.



## DentaLube for Titan® Blis-sonic™ Scalers

For maintenance of all Titan Blis-sonic Scalers K and SW.

**CAUTION: FLAMMABLE. DO NOT USE NEAR HEAT, SPARKS OR FLAME. CONTENTS OF CAN ARE UNDER PRESSURE.**

Do not puncture or incinerate can. Do not expose container to heat or store at temperatures exceeding 130° F (55° C), which can cause can to burst violently and could result in injury.


Use with adequate ventilation. Avoid breathing vapor or spray mist. Avoid contact with eyes and prolonged contact with skin. See SDS for complete health and safety information.

### Keep out of reach of children

#### Do Not Refill

**Instructions:** Lubricate scaler before autoclaving using DentaLube for Titan Blis-sonic Scalers, StarDental® Part #261818.

1. Remove scaler from tubing. Titan Blis-sonic K: Unthread from tubing coupler. Titan Blis-sonic SW: Disconnect from swivel by grasping the scaler with one hand, pulling coupler with other hand. Do not pull on tubing. When separated, swivel remains with coupler.
2. Find drive air tube on Titan Blis-sonic K or notched hole inside of Titan Blis-sonic SW. See illustration.
3. Insert lubricator tip firmly into nozzle. Spray a short burst of DentaLube for Titan Blis-sonic Scalers into tube (Titan Blis-sonic K) or notched hole (Titan Blis-sonic SW).
4. Thread tubing coupler onto Titan Blis-sonic K or snap Titan Blis-sonic SW onto swivel.
5. Attach tip to scaler. Never run scaler without tip in place.
6. Run scaler for 20 seconds to expel excess lubricant; remove tip.
7. Place scaler into paper or plastic autoclave pouch. Autoclave to 135° C.

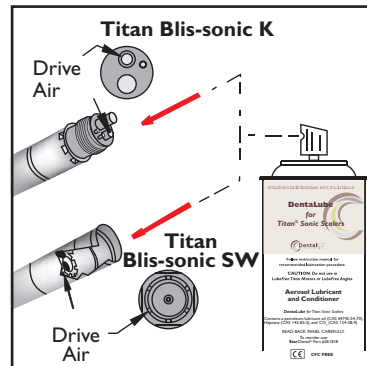

StarDental®  
1816 Colonial Village Lane  
Lancaster, PA 17601 U.S.A.  
Made & Printed in the U.S.A.  
©2019 - DentalEZ, Inc.

624181 Date of Issue 2019/01/10

StarDental® Instruments



California Proposition 65

**WARNING:** Cancer and Reproductive Harm  
www.P65Warnings.ca.gov