


## Titan<sup>®</sup> Angles

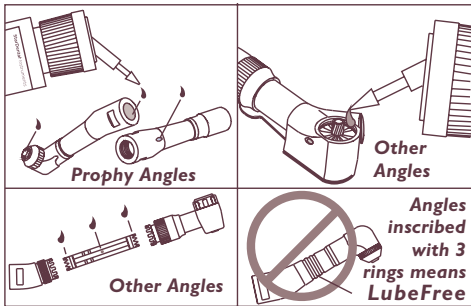

# DentaLube II

## Advanced Handpiece Lubricant



StarDental<sup>®</sup>  
1816 Colonial Village Lane  
Lancaster, PA 17601-5891  
Printed in the USA  
© 2018 – DentalEZ Inc.

Lubricate handpiece prior to each steam sterilization with **Star<sup>®</sup> DentaLube II**.


Place 3 drops of **DentaLube II** into drive air tube.

Run handpiece 20 seconds to distribute lubricant.

**CAUTION:** Do not use in LubeFree High Speed Handpieces, Titan<sup>®</sup> E Attachments or LubeFree Angles.

Part #62560 Rev E 2018/01/15

**StarDental fixed back end lubricated handpieces**

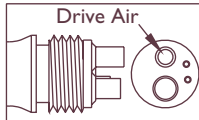

**StarDental swivel style lubricated handpieces except Flex & RA**



**Note:** Never operate handpiece without bur in place.