



LED Lamp for 6-Pin Swivel

The LED Lamp is designed to fit 6-Pin ISO Type C, 3.3 Vdc HiFlo Swivels and designed to connect to **StarBright**[®] Fiber Optic Handpiece Tubing or standard (non-Star) ISO C tubing.

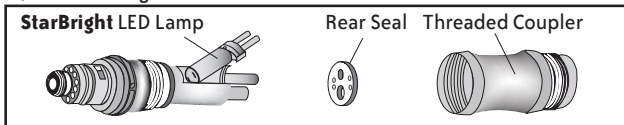


1. Detach rear seal and threaded coupler on **StarBright** Fiber Optic 6-Pin Swivel. Lift lamp up; replace with new, **StarDental** Part #265385. Reassemble threaded coupler and rear seal.

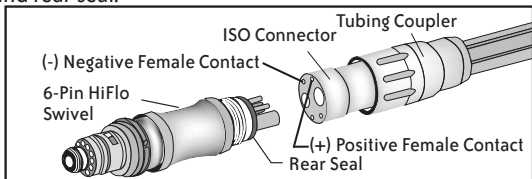

2. Slide tubing coupler back until the end of the ISO connector is exposed. The lamp controller must be putting out 3.3 ± 0.05 volts.
3. Use the digital multimeter to verify that the positive female contact is above the larger hole (exhaust port) at tubing coupler. If polarity is incorrect, then reverse connection of tubing wires at Lamp Control electrical connection ports as shown in illustration below.

Note: Step 3 is not applicable to StarDental Tri-Lamp Controller.

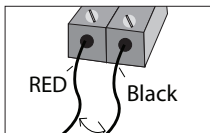

4. Align and insert fittings of the swivel into the ISO connector holes of the tubing. Slide tubing coupler forward and thread clockwise until hand tight against the swivel.
5. Verify that LED swivel operates. If it does not light, immediately shut off and recheck connection and polarity per steps 2 and 3.

StarDental[®] 
1816 Colonial Village Lane
Lancaster, PA 17601 U.S.A.
Printed in the U.S.A.
2018 DentalEZ, Inc.

65440 C Date of Issue: 2018/01/31

866-DTE-INFO
(866-383-4636)
www.dentalez.com