



Alert Symbols The safety notices associated with the following should be given special attention when they appear in maintenance, operating and emergency procedures in the guide.

-  **CAUTION** indicates that a particular procedure or precaution must be followed to avoid system irregularity or possible damage to the product.
-  **NOTE** indicates special information to improve the ease of maintaining the product, or to clarify important information.
-  **WARNING** indicates that the personal safety of the patient, end user or technician could be compromised by disregarding the **WARNING**. Not following instructions may result in an injury.
-  **WARNING:** We recommend consulting a licensed professional, I.E. Structural or Architectural Engineer when attaching a delivery to the building structure. If installing the delivery on a cabinet, shelf or other furniture alternative, take extra precaution to assure the apparatus is structurally sound and secured to the building structure to support the delivery weight preventing detachment or tipping resulting in injury. Forest is not liable for injury resulting from improper installation.
-  **CAUTION:** Field modifications that alter the electrical and/or mechanical safety of Forest products conflict with agency construction file requirements and are not sanctioned by Forest.
-  **CAUTION:** Recommended and provided mounting hardware may not be suitable for your specific installation. Please use caution when selecting hardware to avoid possible damage. Forest is not liable for damage due to hardware choice.
-  **NOTE:** Installation Verification: Refer to Operator's Guide or Service Manual on our website at Forestdental.com; perform all operations. If operations performing as intended installation complete. If operations not performing as intended, review steps in installation instructions to confirm correct installation or call Forest Technical Support at 800-423-3555.


Tools Required: 1/8", 7/32" and 5/16" Allen Wrench, and Bubble Level.

Beneath Counter Mount


Mounting Hardware Included: 3/8-16 x 7/8" Socket Button Head Grade 8 Screws, 4 each.

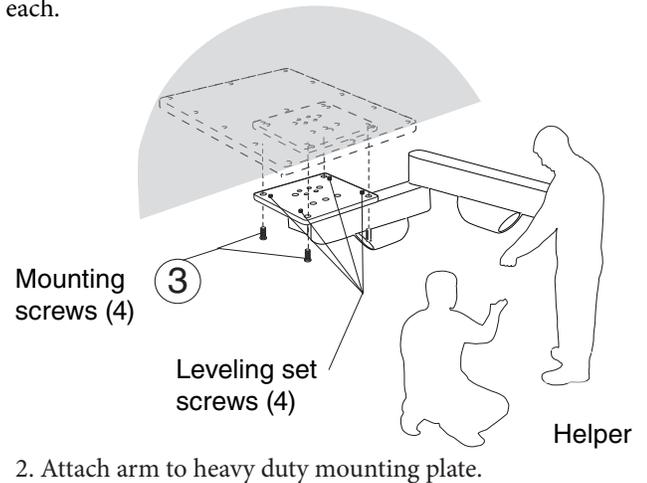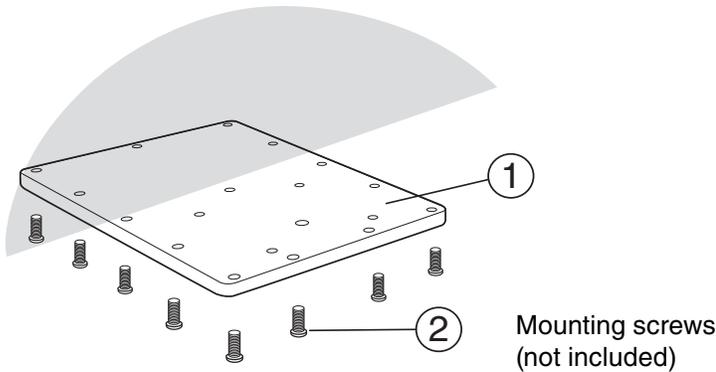

Beneath Counter Plate, 11" x 14.5"

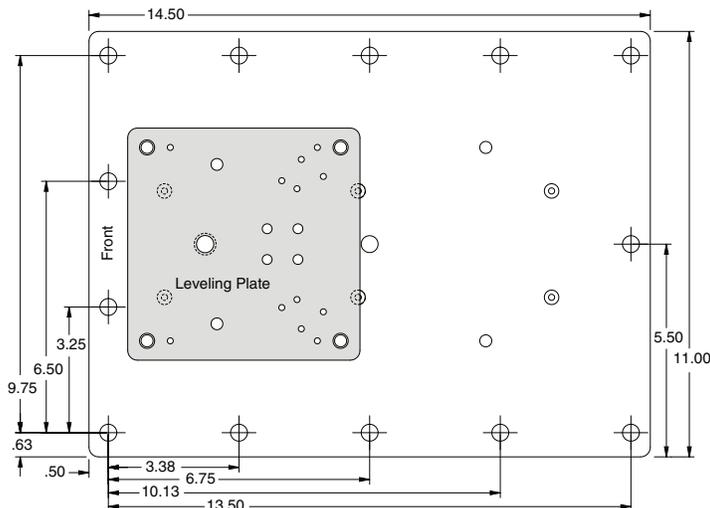



 **NOTE:** Not to scale.

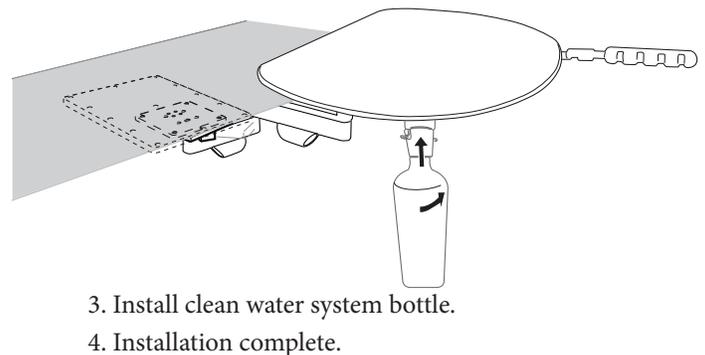

Wall Mount

Mounting Hardware Included: 3/8-16 x 1-1/4" Socket Button Head Screws, 4 each.

NOTE: A 20" mounting height from the floor will result in the unit work surface having a height of 34" from the floor. Check with the operator to ensure the installation is acceptable.

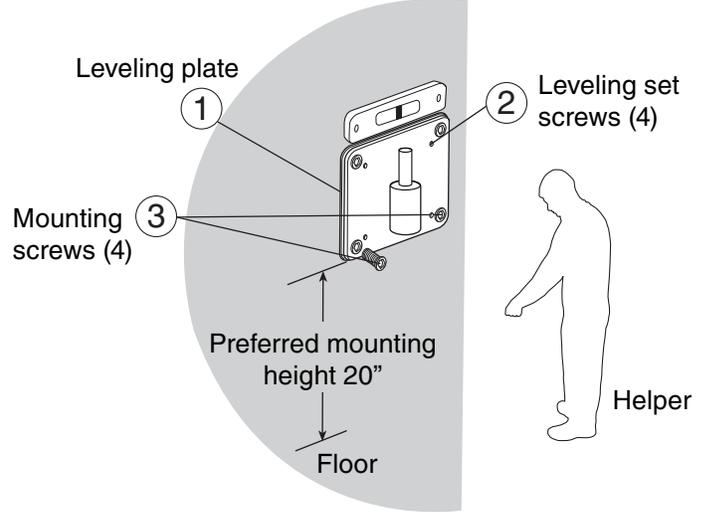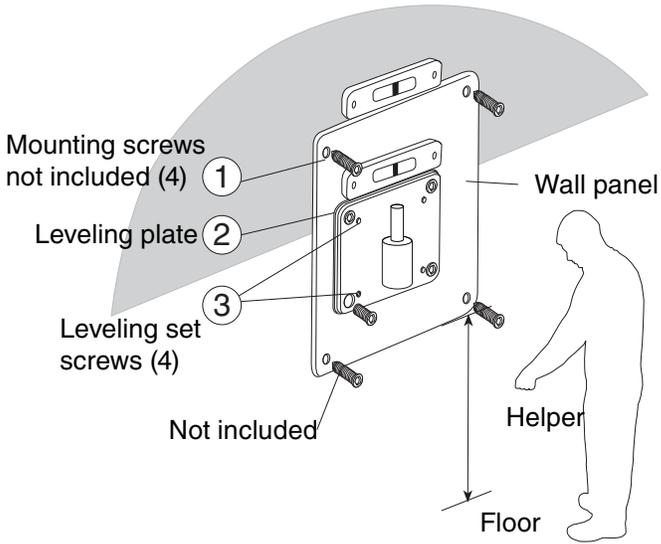

1. Mount large 16" x 18" plate on wall. If wall mount plate is not needed, proceed to step 2.




2. Attach 6" x 6" mounting plate.
3. Install arm knuckle into plate hub.

NOTE: For control head leveling refer to page 1.

Mounting Plate, 6" x 6"

Wall Mount Plate

Metal powder coated, 16" x 18"
Laminate, 18" x 23.75"

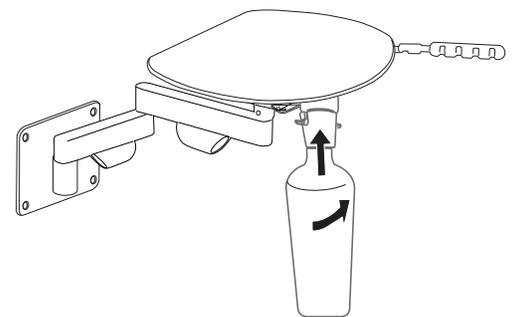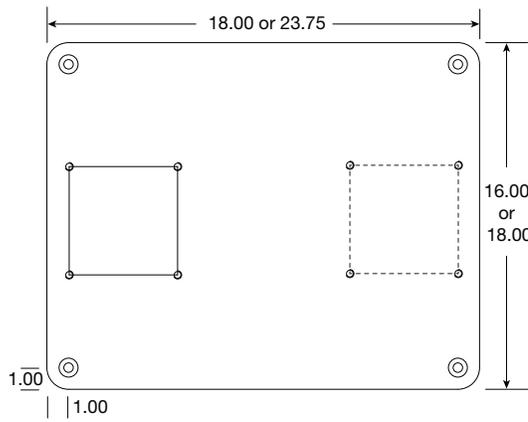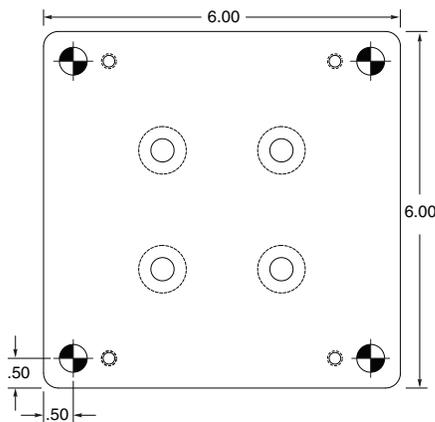

NOTE: Not to scale.

NOTE: Not to scale.

4. Install clean water system bottle.
5. Installation complete.



DTE Oregon, Inc.
Address 6200 ne cherry drive hillsboro oregon 97124 • USA

Phone 800 . 423 . 3555 • 503 . 640 . 3012 | Fax 503 . 693 . 9715 | Digital sales@forestdental.com • forestdental.com

