

### Assistant's (Figure 1)

All controls are located underneath the work surface.

### Duo or Doctor's (refer to 0097-331 for control head operation)

All controls are located on the control head.

### Optional Quick Disconnect (Figure 2)

Connection for additional water and/or air source.

### Air Flow Adjustment For Quick Disconnect

Allows adjustment of air through the quick disconnect. Turn knob counter clockwise to increase flow, clockwise to decrease flow.

### Water Flow Adjustment For Quick Disconnect

Allows adjustment of water through the quick disconnect. Turn knob counter clockwise to increase flow, clockwise to decrease flow.

Figure 1. Controls Underneath the Work Surface

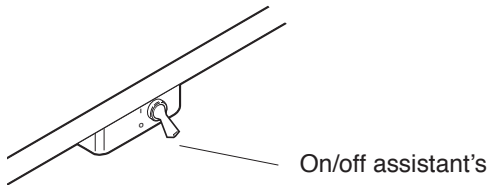


Figure 2. QD Controls Underneath the Work Surface (option not standard)


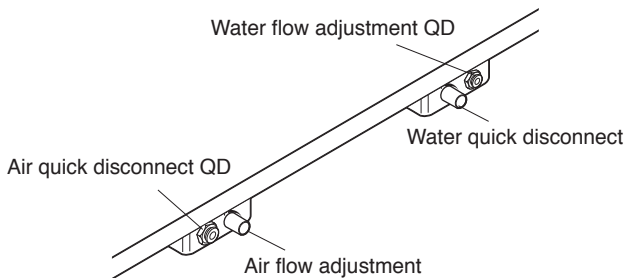


Figure 5: Height Adjustment

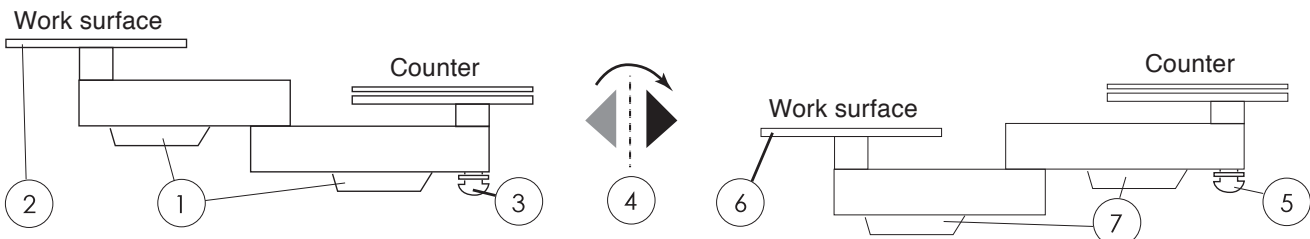

### Leveling Adjustment (Wall Mount) (Figure 3)

Adjust (4) leveling set screws to level.

### Leveling Adjustment (Beneath Counter Mount) (Figure 4)

Adjust (4) leveling set screws to level.

### Drift Set Screws (Figure 4)

Loosen or tighten (3) drift set screws.

### Height Adjustment (Lowering) (Figure 5)

1. Remove umbilical cuffs/holders.
2. Lift work surface off arm.
3. Loosen and remove washer & nut.
4. Flip arm assembly (see illustration below).
5. Reinstall arm (by tightening the washer & nut).
6. Reattach work surface back on arm.
7. Route umbilical and reinstall umbilical cuffs/holders.

Figure 3: Wall Mount


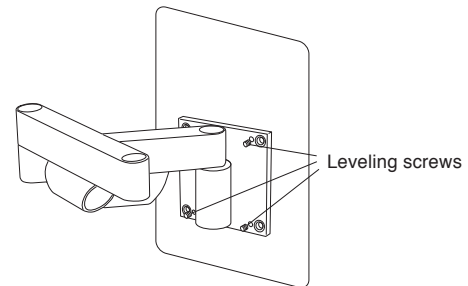


Figure 4: Beneath Counter





Forest Dental Equipment

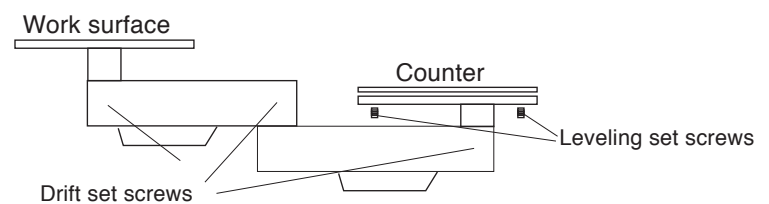

DTE Oregon, Inc.  
Address 6200 ne cherry drive hillsboro oregon 97124 • USA

---

Phone 800 . 423 . 3555 • 503 . 640 . 3012 | Fax 503 . 693 . 9715 | Digital sales@forestdental.com • forestdental.com

CE 0459

4.13.19 • 0097-418 • Rev. E • 2 of 2