



## BioFree Tubing™

All DentalEZ Delivery Systems come standard with BioFree tubing on all internal water lines and handpiece tubings. The BioFree tubing has been physically imbedded with silver ions.

BioFree tubing can remain effective for several years without maintenance however it must be regularly maintained. We recommend Sterisil® waterline treatment products to regenerate the silver ions. When properly maintained, the BioFree tubing will remain effective for the normal life of the dental unit. CitriSil™ tablets keep dental water safe by maintaining clean dental unit water lines.

### Product Features

- Suppresses the growth of bacteria, algae, fungus, mold and mildew
- Minimizes the need for flushing and the use of corrosive chemicals
- Reduces line clogging, water flow restrictions and sticky handpieces




## Clean Water System

Operation of the clean water system is as follows:

- Flip the toggle switch **(A)** to the **ON** position.
- Flip the toggle switch **(B)** to change usage from bottle water to city water.
- Check the gauge **(C)** pressure. It should read 40 PSI.
- To adjust water pressure, turn the set screw **(D)** on the regulator located on the underside of the manifold.
- To remove the bottle **(E)**, first flip the toggle switch **(A)** to the **OFF** position, and then unscrew it from the assembly.

Empty the bottle at the end of each day. Then before the day's first procedure, fill it with fresh clean water and follow the Citrisil Maintenance guidelines.

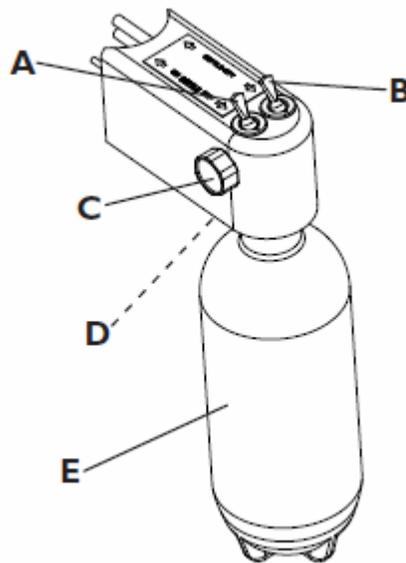



## Citrisil

Once a month use CitriSil's shock oxidizer and enhanced mineral deposit cleaner, an effervescent orange tablet to remove mineral related waterline deposits. Then with each self-contained water bottle refill, simply add the maintenance tablet.

As the tablet dissolves, Sterisil's patented formulation is released, which automatically removes mineral related deposits within dental unit water lines.

CitriSil™ is safe for patients and clinicians, and won't harm dental equipment. It does not produce odor or taste and does not adversely affect composite bond strength.

CitriSil™ also maintains Bio-Free™ tubing and bottle products installed in many new dental units.

