



Galaxy/Simplicity Delivery Unit Cleaning/Disinfecting

Dental Unit:

The Simplicity Magellan unit should be cleaned as follows:

- Wipe frequently with a damp, lint-free cloth.
- Use a mild detergent to clean stains. **(Do not use abrasive cleaning agents)**
- Liquid disinfectants are recommended for use on all Simplicity Magellan external surfaces and should be applied using a soft, clean cloth.
- Cavicide™ is the recommended disinfectant for use on all external surfaces.

NOTE: Avoid using disinfectants in spray containers because they may cause premature staining, discoloration and/or damage to the unit

- At the end of the day, turn the master **ON/OFF** valve to the **OFF** position and clean the instrumentation.
- Each morning, turn the master valve **ON** and check the air and water system for proper operation using the three-way syringe.

Solids Collector

To clean the solids collector, located at the instrument end of the assistant's arm, do the following:

1. Push the lever on the saliva ejector downward to equalize pressure.
2. Remove the lid by lifting it straight up.
3. Grasp the stem on the basket and lift the basket straight up.
4. Empty debris and wash out the basket with warm water, then replace the basket and lid.

See specific cleaning and sterilization instructions for High Volume Evacuator (HVE), Saliva Ejectors (SE), Air/Water Syringe, Clean Water System and Bio-Free Tubing.

CAUTION

Do not use disinfectants that contain any of the following:

- Iodophors
- Glutaraldehydes
- Phenols
- Sodium Hypochlorites

