

! NOTE: Mounting screws installed with cuspidor hub, canister and decorative cover during clean water system installation if cuspidor was ordered with an FE model.

STEP 1: Install cuspidor clamp assembly onto control post (FIG 1).

STEP 2: Insert the cuspidor bushing into hub (FIG. 2).

STEP 3: Insert cuspidor arm into bushing; route drain tubing out the bottom and through the hole in the Forest flat chair adapter.

STEP 4: Cover the drain tubing with spiral flex tubing and secure at the utility center.

! NOTE: The cuspidor umbilical runs separate from the doctors unit umbilical.

STEP 5: Install the cuspidor bowl, strainer, spouts, and cup holder.


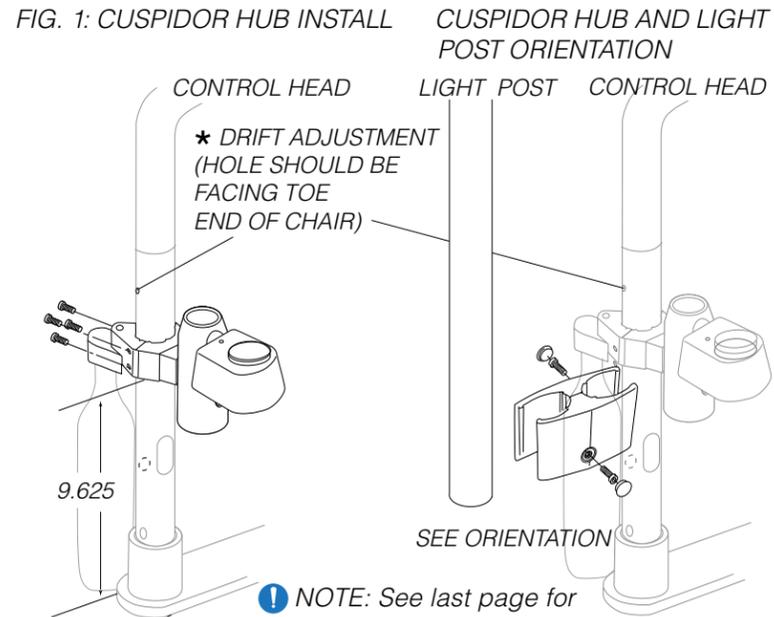


FIG. 2:

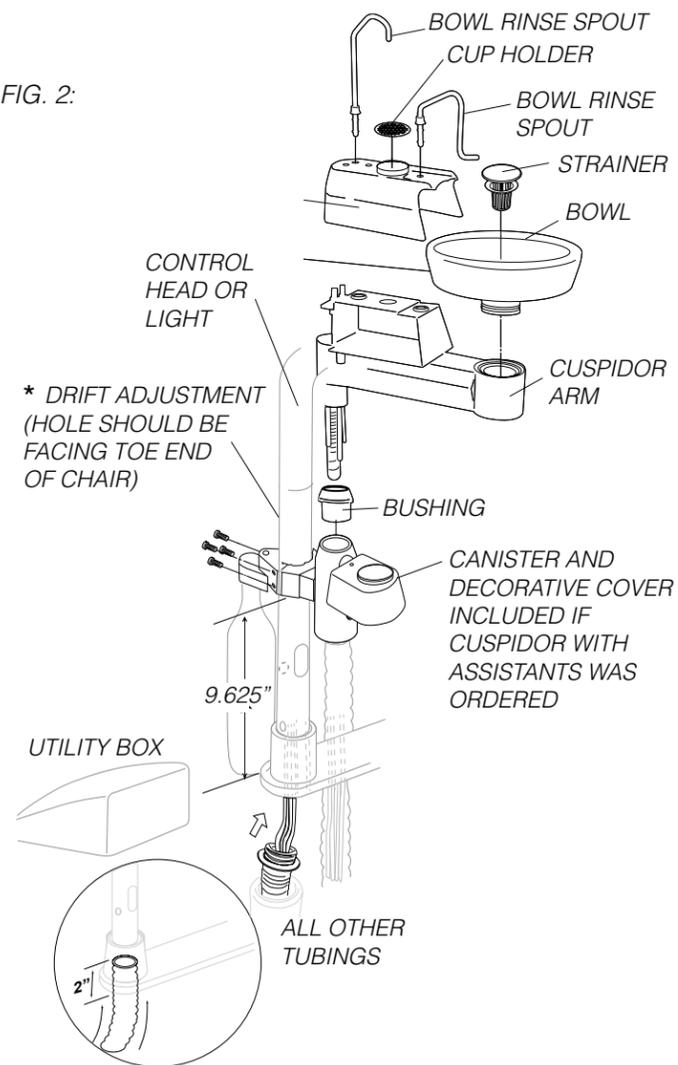

Forest DENTALEZ

ALERT SYMBOL KEY

The safety notices associated with the following should be given special attention when they appear in maintenance, operating and emergency procedures in the guide.

! WARNING indicates that the personal safety of the patient, end user or technician could be compromised by disregarding the WARNING. Not following instructions may result in an injury.

! CAUTION indicates that a particular procedure or precaution must be followed to avoid system irregularity or possible damage to the product.

! NOTE indicates special information to improve the ease of maintaining the product, or to clarify important information.

! CAUTION: Field modifications that alter the electrical and/or mechanical safety of Forest™ products conflict with agency construction file requirements and are not sanctioned by Forest.

! NOTE: Installation Verification: Refer to Operator's Guide and perform all operations. If operations performing as intended installation complete. If operations not performing as intended, review steps in installation instructions to confirm correct installation or call Technical Support at 800-423-3555.

Tools Required: 5/32", 3/16", 5/64", 1/8" Hex Key and Bubble Level.

! NOTE: If installing a monitor assembly, refer to the Monitor Mount Installation Instruction.

LIGHT ONLY NO CONTROL HEAD


STEP 1: Insert light post into chair adapter hub (FIG. 1).

STEP 2: Level post using a bubble level and the leveling screws in the chair adapter hub. Tighten screws securely.

STEP 3: Insert light bushing in the post.

STEP 4: Insert light into bushing while routing the light cable down through the post and out the bottom to the power supply.

FIG. 1:

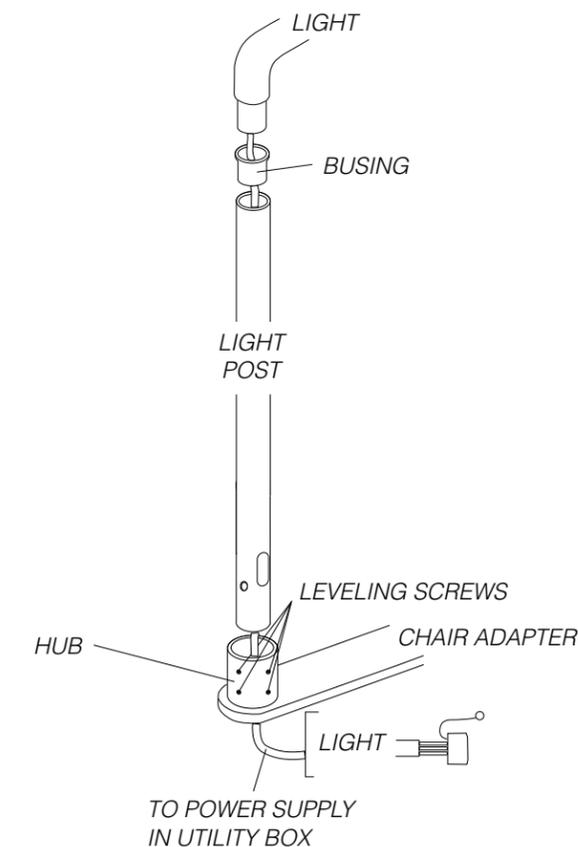

DOCTORS DELIVERY UNIT

NOTE: For umbilical connection refer to the utility center installation instruction and schematic.


STEP 1: Insert unit control post into chair adapter hub (FIG. 1).

STEP 2: Level post using a bubble level and the leveling screws in the chair adapter hub. Tighten screws securely.

STEP 3: Insert the control head arm into the post proceeded by the nylon washer and trim ring.

STEP 4: Route unit umbilical through the control post to the utility center.

FIG. 1:

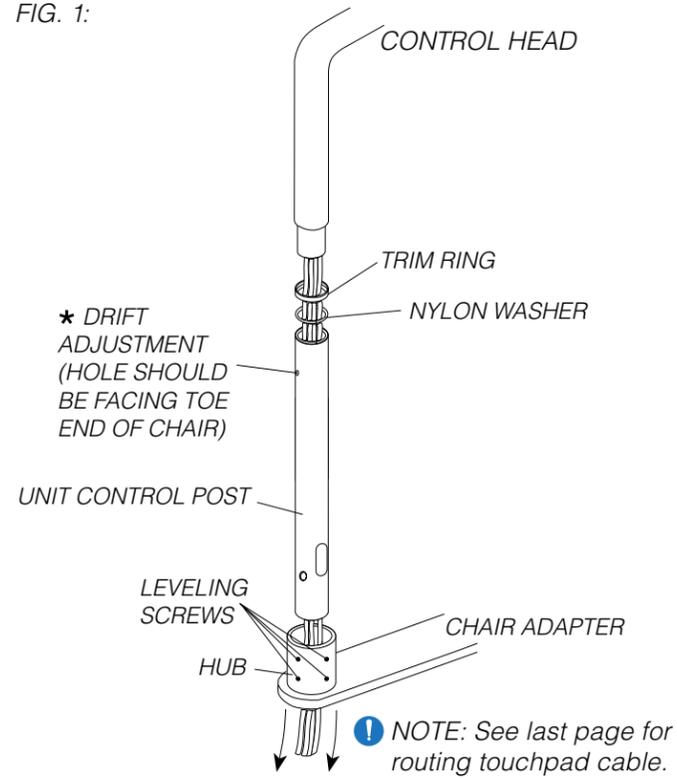

CLEAN WATER SYSTEM

NOTE: If light post was ordered, install the light post clamp prior to the clean water clamp.

STEP 1: Install the clean water system onto the unit control post (FIG. 2).

NOTE: Clean water system & hole should be facing toe end of chair.


STEP 2: Route the tubing through and down the unit control post to the utility center.

NOTE: If the unit-mounted touchpad option was ordered, route the cable under the chair adapter to the chair board and connect.

STEP 3: Cover the umbilical with spiral flex tubing and secure using the umbilical ring under the post and cuffs at the utility center.

NOTE: If a light and/or assistants telescoping arm was ordered, skip step 3 and proceed to page 3.

FIG. 2:

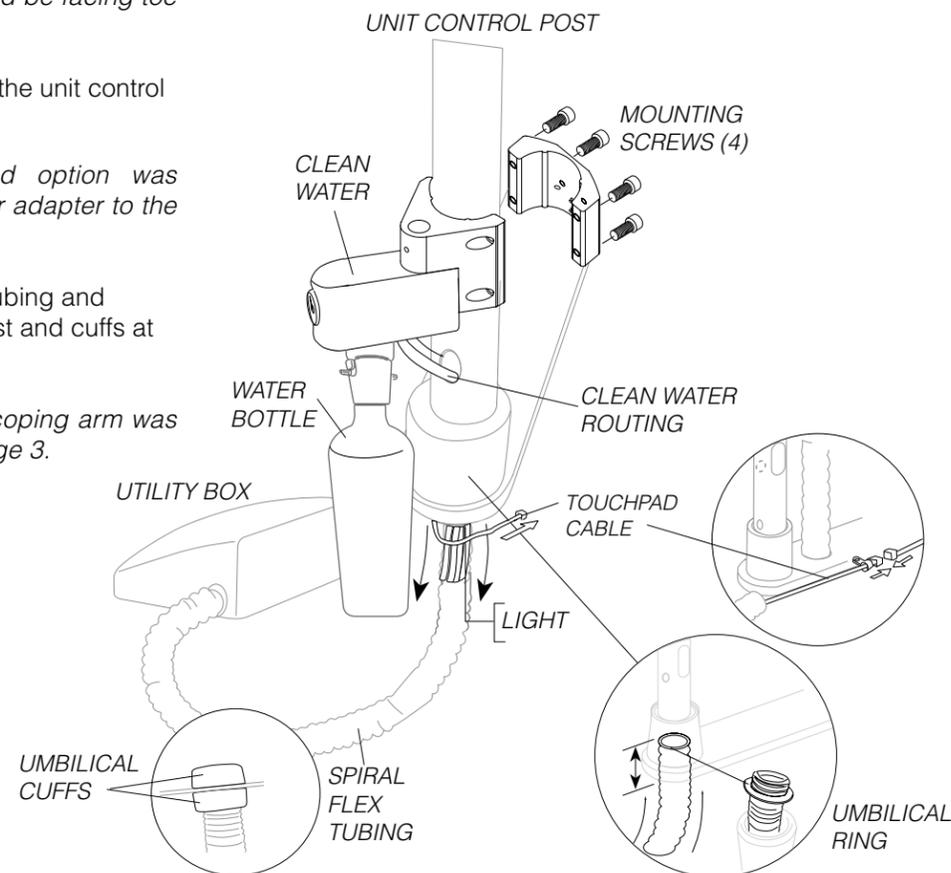

LIGHT POST ASSEMBLY

STEP 1: Install the light post and clamp assembly onto the unit control post using the orientation illustration for configuration reference. Tighten screws securely (FIG. 1).

STEP 2: Insert light bushing in the post.

STEP 3: Insert light into bushing while routing the light cable down through the post and out the bottom as well as into the unit control post and out the bottom.

STEP 4: Proceed to step 3 under the clean water system section on page 2 to conceal unit umbilical and light cable if assistants telescoping arm was not ordered.

NOTE: If light and post assembly were ordered without doctors delivery unit, spiral flex tubing is not included.

NOTE: If an assistants telescoping arm was ordered, skip step 4 and proceed to assistants telescoping arm below.

FIG. 1:

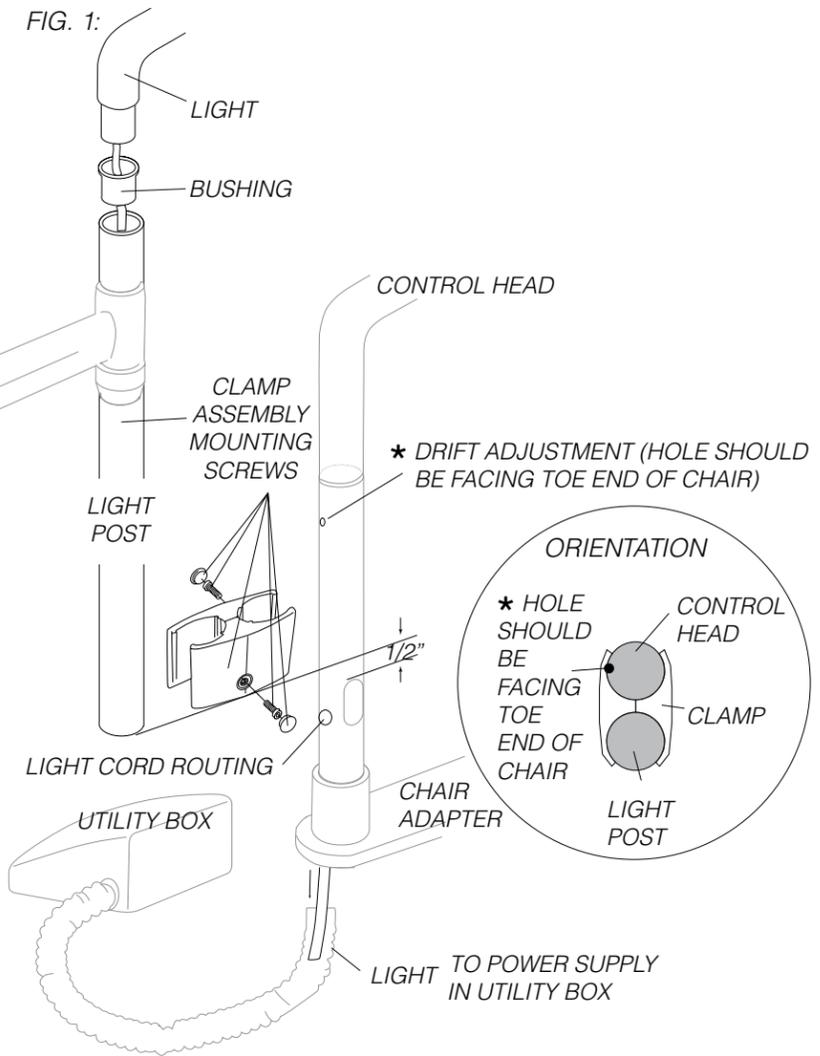


FIG. 1:

ASSISTANT'S TELESCOPING ARM

NOTE: Mounting screws installed with cuspidor hub, canister and decorative cover during clean water system installation if cuspidor was ordered with an FE model.

STEP 1: Insert telescoping arm links 1/2" pin into hole on top of clamp assembly (FIG. 1).

STEP 2: Route the drain tubing into the unit control post through the slot and out the bottom of post.

STEP 3: Attach HVE(s) and saliva ejector tubing to canister ports under the decorative cover.

STEP 4: Hang Syringe, HVE(s) and saliva ejector in the holder.

NOTE: Level telescoping arm if drifting occurs.

STEP 5: Proceed to step 3 under the clean water system section to conceal the unit umbilical, drain tubing and light cable (if applicable).

NOTE: If assistants telescoping arm was ordered without doctors delivery unit, skip step 5 and route drain tubing to the utility center.

