CEILING MOUNT INSTALLATION

Tools Required: 5/64", 1/8", 5/32", 1/4" Allen Wrench, 3/16" & 11/32" Drill Bit, Electric Drill Motor, 1-1/2" Hole Saw, Phillips Screwdriver, 7/16" Socket Ratchet, Bubble Level.

STEP 1: Position ceiling plate and drill holes using measurements from page 2 and 3 taking into consideration ceiling type from page 1.

STEP 2: Pull line voltage down through plate (FIG. 1)

STEP 3: Install lag bolts into required backing.

STEP 4: Make line voltage connections. This step must be done by a licensed electrician (FIG. 2).

STEP 5: Connect power supply to wire harness.

STEP 6: Install low voltage wire through slot and pull down through drop post hub for pigtail connection (FIG. 3).

STEP 7: Plug in pigtail connectors (FIG. 4).

STEP 8: Remove main support screw and insert drop post into hub while aligning the main support screw hole in post with the hole in hub to ensure proper installation of main support screw. Install main support screw through hub into post. Main support screw head will be flush with hub when properly installed.

⚠ WARNING: Ensure main support screw passes thru hole in hub and post. Screw head will be flush with hub when properly installed. Drop post will hang securely when main support screw is properly installed. Once secured, adjust leveling screws.

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STEP 9: Use level to plumb post by adjusting leveling screws.

STEP 10: Slide decorative cover over post to conceal ceiling plate and components (FIG. 5).

STEP 11: Install cover ring and set screw firmly against post to secure in place.

STEP 12: Make pigtail connections (FIG. 6).

STEP 13: With swivel key inserted into swivel sleeve slot, slide swivel sleeve onto rigid light arm.

STEP 14: Insert rigid arm into drop post and install screws through post and swivel sleeve.

STEP 15: Rigid arm hub groove contains an anti-rotation locking set screw installed. If reverse rotation is required, relocate set screw to hole on other side of rigid arm hub groove. When reinserting key, be sure key position is away from headrest.

FIG. 1:

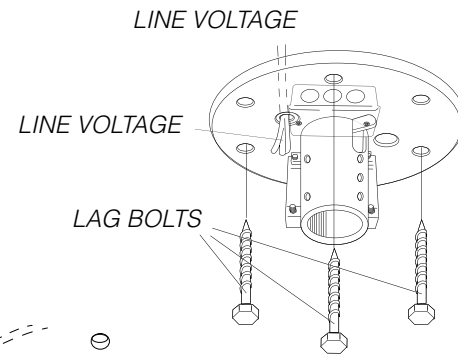
FIG. 2:

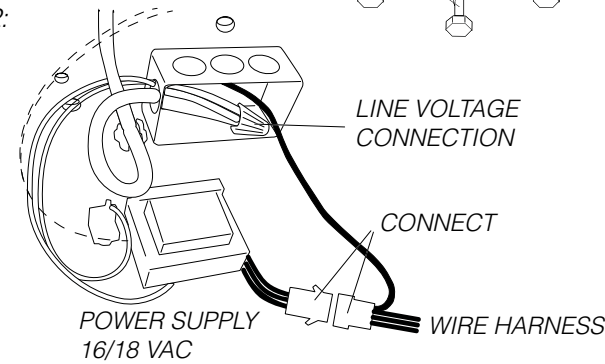
FIG. 3:

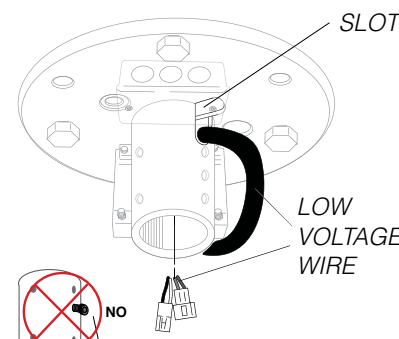
FIG. 4:

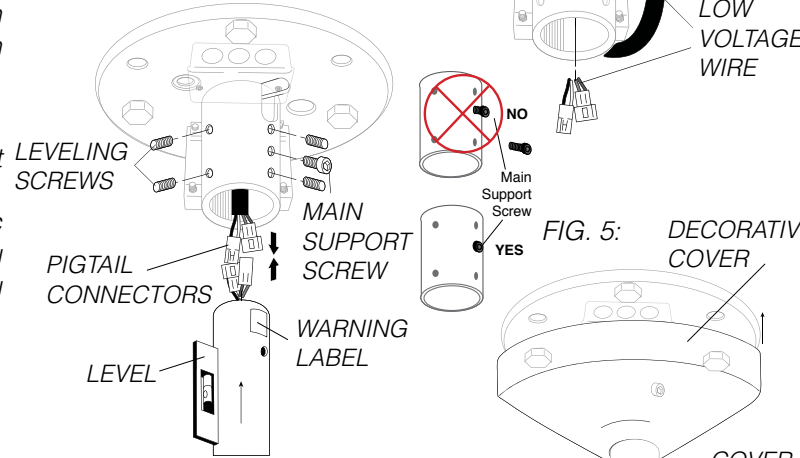
FIG. 5:


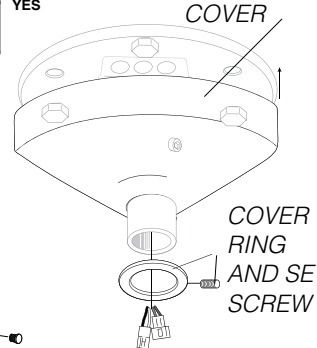
FIG. 6:

CEILING MOUNT OPERATORY LIGHT

Forest DENTALEZ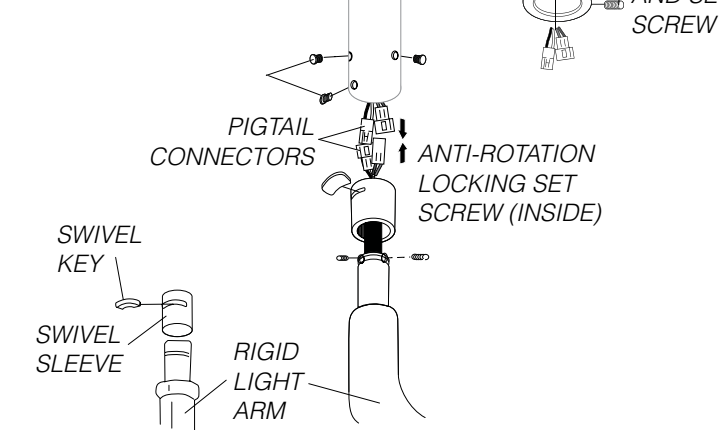

Refer to Forest™ General Information Operator's Guide for Glossary of Symbols, Terms, EMC/Electrical Safety Declaration and User Guidance.

⚠ WARNING: We recommend consulting a licensed professional, i.e. Structural or Architectural Engineer when attaching a fixture to the building structure.

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⚠ WARNING: To ensure proper installation, only authorized technicians should install this product.

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⚠ WARNING: Ensure power supply/transformer is located inside a tool-accessible covered box and away from water sources.

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⚠ CAUTION: Field modifications that alter the electrical and/or mechanical safety of Forest products conflict with agency construction file requirements and are not sanctioned by Forest.

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⚠ CAUTION: Recommended and provided mounting hardware may not be suitable for your specific installation due to ceiling type. Please use caution when selecting hardware due to supporting structure type to avoid possible damage. Forest is not liable for damage due to hardware choice.

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INSTALLATION INSTRUCTION

⚠ CAUTION: Ceiling and bracing type is shown as "reference only".

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ⓘ NOTE: Installation Verification: Refer to Operator's Guide and perform all operations. If operations performing as intended installation complete. If operations not performing as intended, review steps in installation instructions to confirm correct installation or call Technical Support at 800-423-3555.

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