

COMFORT STOOL

Forest

DENTALEZ

INSTALLATION INSTRUCTION

ALERT SYMBOL KEY

The safety notices associated with the following should be given special attention when they appear in maintenance, operating and emergency procedures in the guide.

⚠ WARNING indicates that the personal safety of the patient, end user or technician could be compromised by disregarding the WARNING. Not following instructions may result in an injury.

⚠ CAUTION indicates that a particular procedure or precaution must be followed to avoid system irregularity or possible damage to the product.





! NOTE indicates special information to improve the ease of maintaining the product, or to clarify important information.

⚠ WARNING: Only engage piston inside the height adjustment cylinder after the stool is assembled. Activating the piston prior to installation may cause injury.

! NOTE: Installation Verification: Refer to Operator's Guide and perform all operations. If operations performing as intended installation complete. If operations not performing as intended, review steps in installation instructions to confirm correct installation or call Technical Support at 800-423-3555.

6176 DOCTOR'S COMFORT STOOL

STEP 1: Insert casters (wheels) into base. Apply a significant amount of pressure to ensure casters are seated properly (FIG. 1-2).

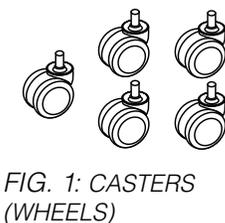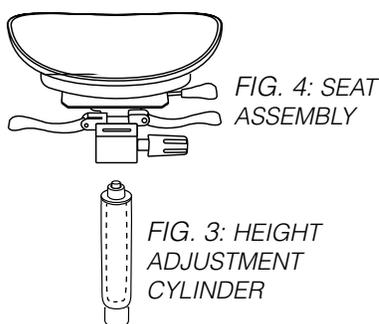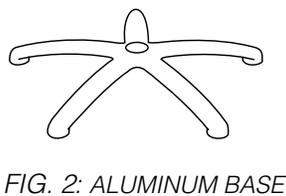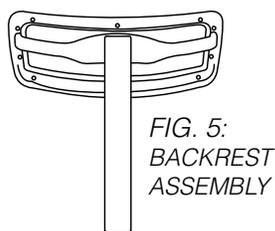


STEP 2: Insert height adjustment cylinder into aluminum base (FIG. 3).

STEP 3: Place the seat assembly on top of height adjustment cylinder (FIG. 4).

STEP 4: Remove black plug and insert back assembly into seat assembly. Replace black plug after installation (FIG. 5).

STEP 5: Sit on stool to permanently lock cylinder into base and seat. At this time, the piston in the height adjustment cylinder may be engaged by activating the seat height lever to adjust seat location.

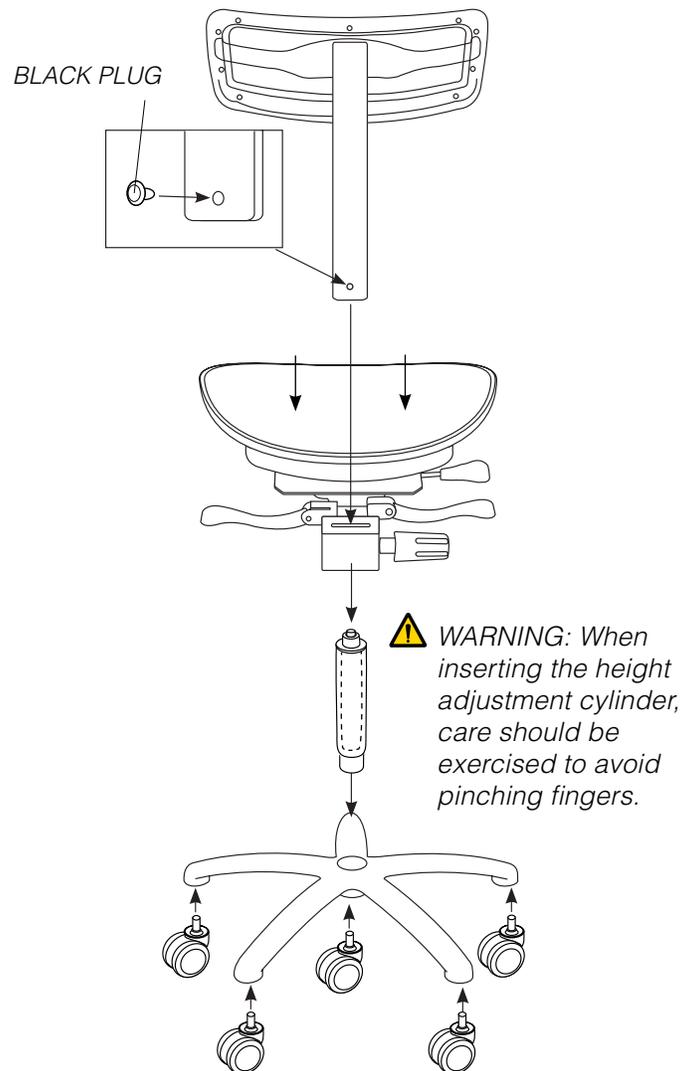

⚠ WARNING: Be sure casters are firmly seated before using stool.

6187 ASSISTANT'S COMFORT STOOL

STEP 1: Insert casters (wheels) into base. Apply a significant amount of pressure to ensure casters are seated properly (FIG. 1-2).

STEP 2: Insert height adjustment cylinder into base (FIG. 3).

STEP 3: Slide adjustable footring over height adjustment cylinder and tighten footring knob to secure in place (FIG. 4).

STEP 4: Slide chrome arm tube on top of height adjustment cylinder until firmly seated (FIG. 5).

STEP 5: Place the seat assembly on top of height adjustment cylinder (FIG. 6).

STEP 6: Remove black plug and insert back assembly into seat assembly. Replace black plug after installation (FIG. 7).

STEP 7: Slide ratcheting arm into arm tube. Firmly tighten arm tube knob clockwise to secure arm in place (FIG. 8).

STEP 8: Sit on stool to permanently lock cylinder into base and seat. At this time, the piston in the height adjustment cylinder may be engaged by activating the seat height lever to adjust seat location.

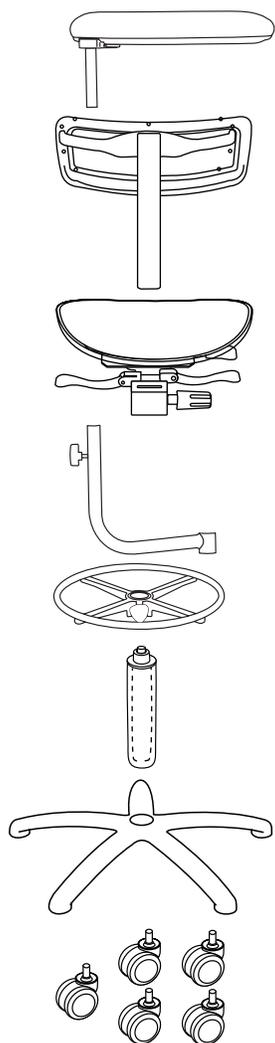


FIG. 8: RATCHETING ARM

FIG. 7: BACKREST ASSEMBLY

FIG. 6: SEAT ASSEMBLY

FIG. 5: CHROME ARM TUBE

FIG. 4: ADJUSTABLE FOOTRING

FIG. 3: HEIGHT ADJUSTMENT CYLINDER

FIG. 2: ALUMINUM BASE

FIG. 1: CASTERS (WHEELS)

