


# Forest


## DENTALEZ

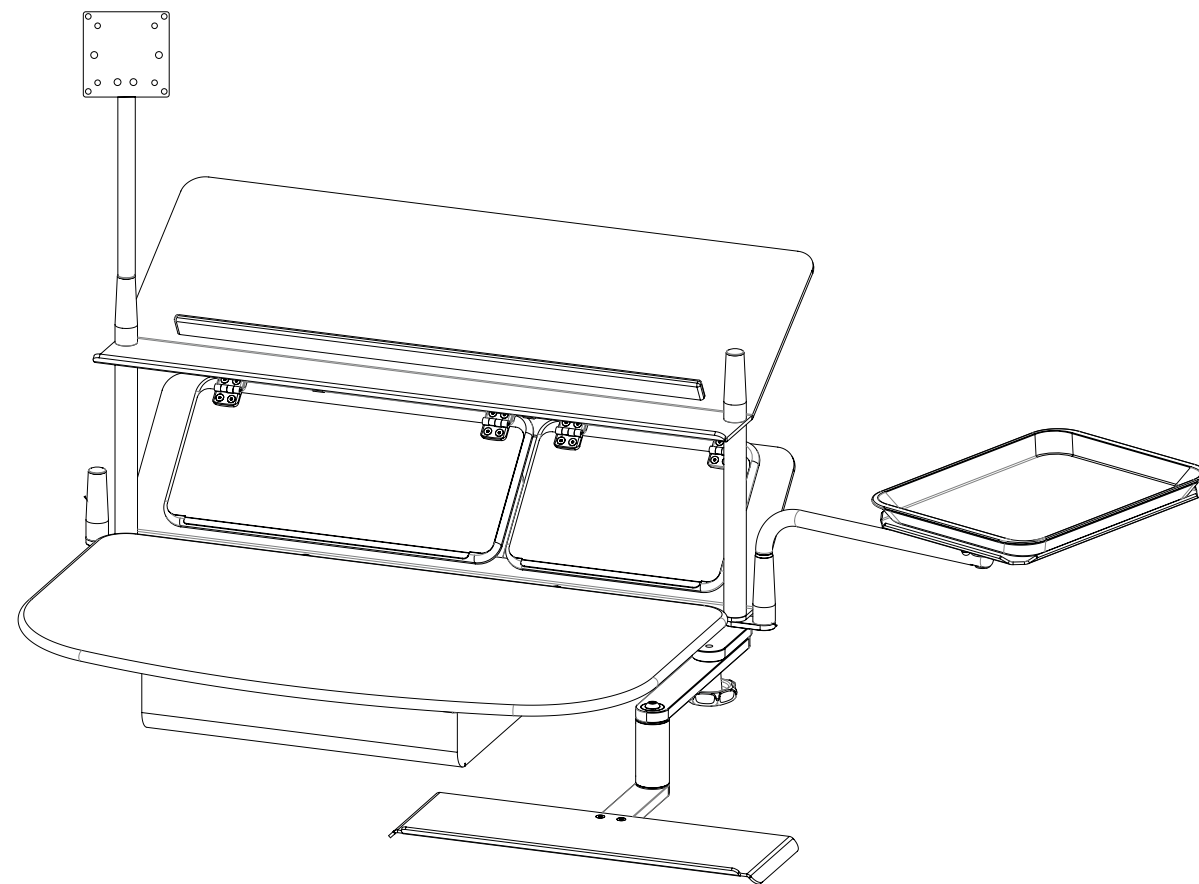

### CPU/TRASH SHELF

STEP 1: Attach shelf to H-frame.

CPU/trash shelf installed.

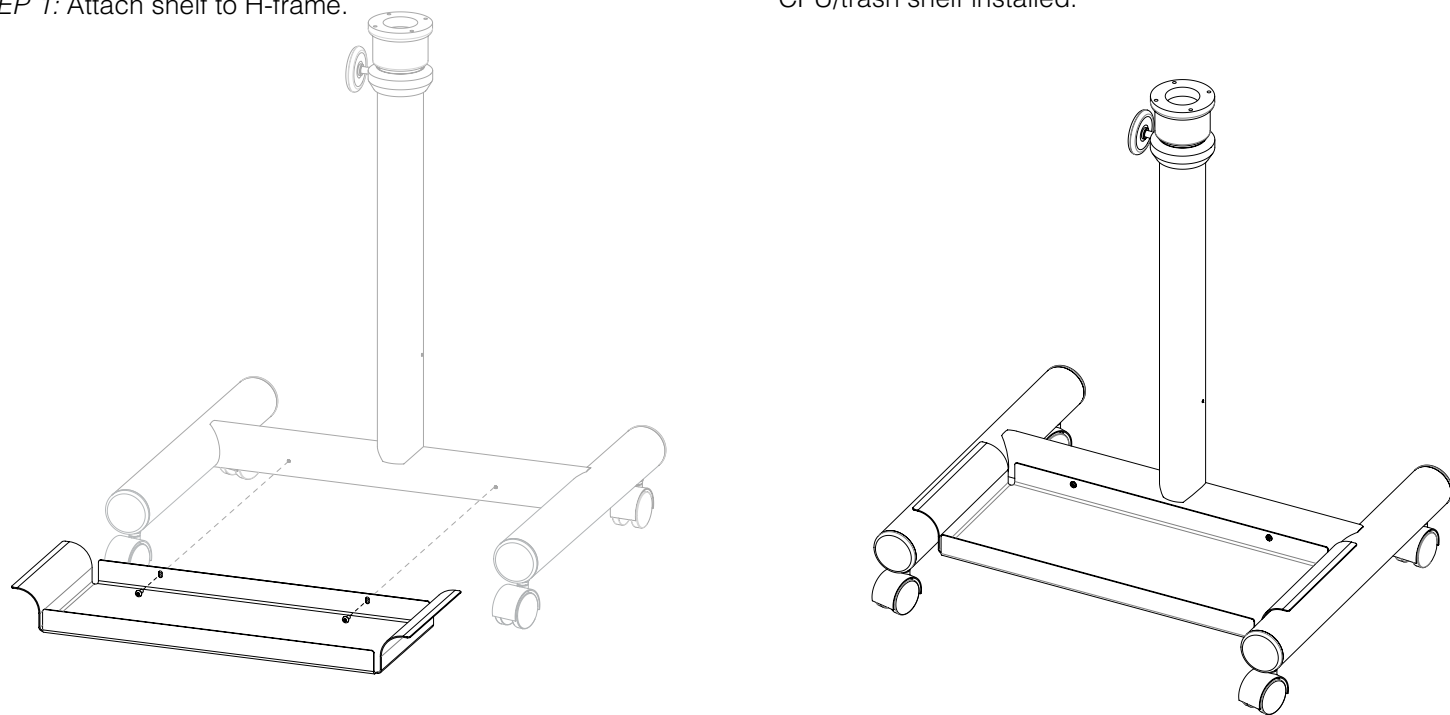

### ALERT SYMBOL KEY

The safety notices associated with the following should be given special attention when they appear in maintenance, operating and emergency procedures in the guide.

**⚠ WARNING** indicates that the personal safety of the patient, end user or technician could be compromised by disregarding the WARNING. Not following instructions may result in an injury.

**⚠ CAUTION** indicates that a particular procedure or precaution must be followed to avoid system irregularity or possible damage to the product.

**📌 NOTE** indicates special information to improve the ease of maintaining the product, or to clarify important information.

**⚠ WARNING:** We recommend consulting a licensed professional, I.E. Structural or Architectural Engineer when attaching a delivery to the building structure. If installing the delivery on a cabinet, shelf or other furniture alternative, take extra precaution to assure the apparatus is structurally sound and secured to the building structure to support the delivery weight preventing detachment or tipping resulting in injury. Forest™ is not liable for injury resulting from improper installation.

**⚠ CAUTION:** Field modifications that alter the electrical and/or mechanical safety of Forest™ products conflict with agency construction file requirements and are not sanctioned by Forest.

**⚠ CAUTION:** Recommended and provided mounting hardware may not be suitable for your specific installation. Please use caution when selecting hardware to avoid possible damage. Forest is not liable for damage due to hardware choice.

**📌 NOTE:** Installation Verification: Refer to Operator's Guide and perform all operations. If operations performing as intended installation complete. If operations not performing as intended, review steps in installation instructions to confirm correct installation or call Technical Support at 800-423-3555.

Tools Required: 1/8" and 7/32" Allen Wrench, 9/16" Ratchet.

**📌 NOTE:** Some parts may have been installed by Forest.


### ORGANIZER BINS, TRAYS AND MONITOR ASSEMBLY

STEP 1: Attach organizer brackets (FIG. 1-1.1).

STEP 2: Attach towers to brackets.


### WORK SURFACE

FIG. 1:

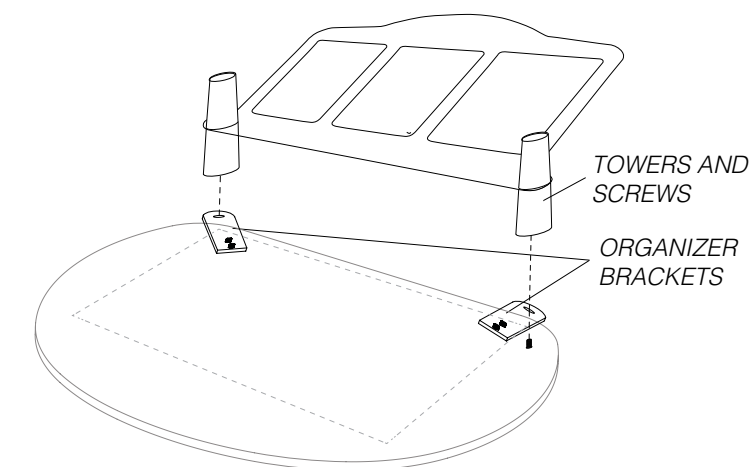

### 7000 SERIES DELIVERY

FIG. 1.1:

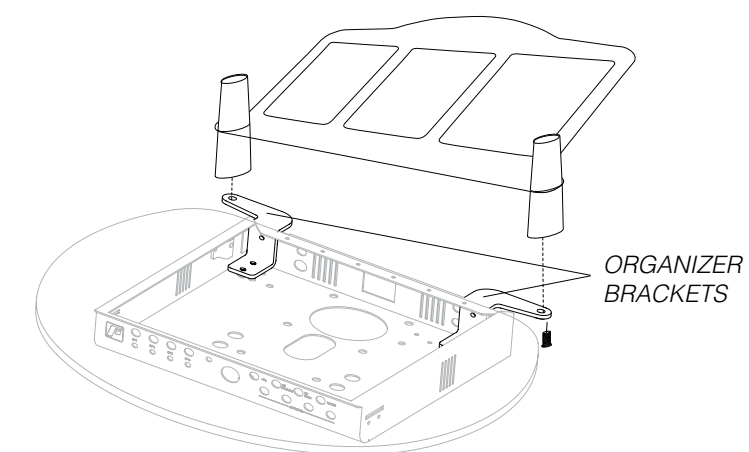

**ORGANIZER BINS, TRAYS AND MONITOR ASSEMBLY (CONTINUED)**

**WORK SURFACE AND 7000 SERIES DELIVERY**

**STEP 3:** Install bins into organizer cut outs (FIG. 2).

**STEP 4:** Remove tower plug and insert tray (FIG. 3).

**STEP 5:** Remove tower plug and insert monitor mount (FIG. 4).

**STEP 6:** Final assembly (FIG. 5).

FIG. 2:

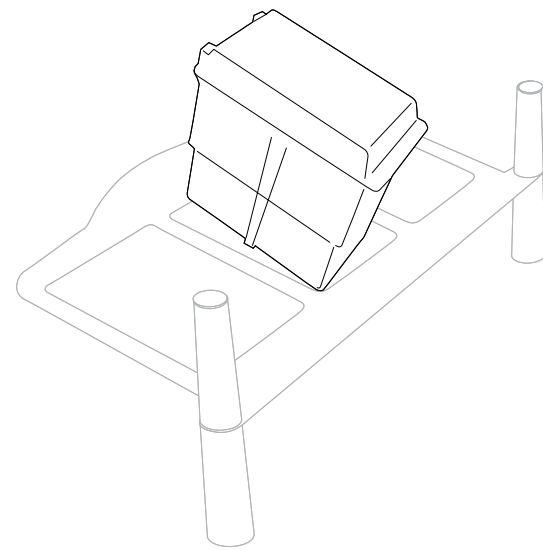


FIG. 3:

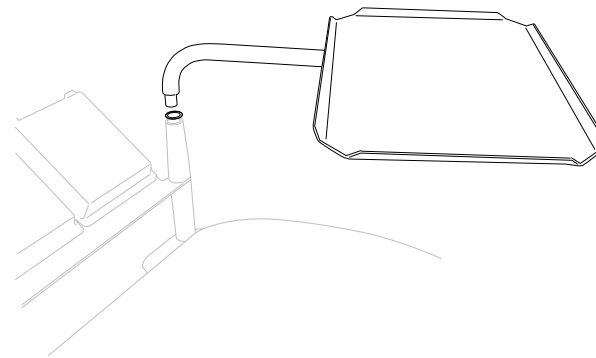


FIG. 4:


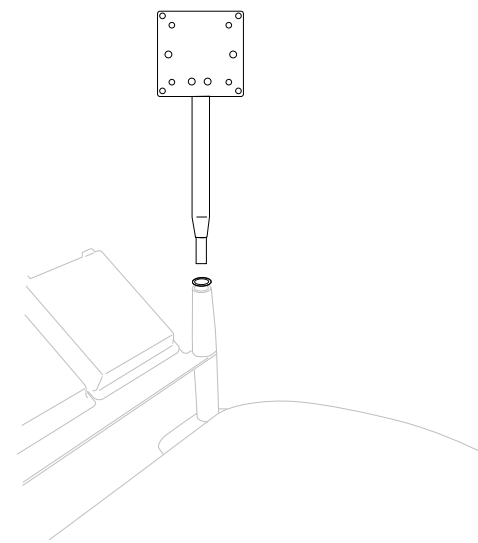


FIG. 5:

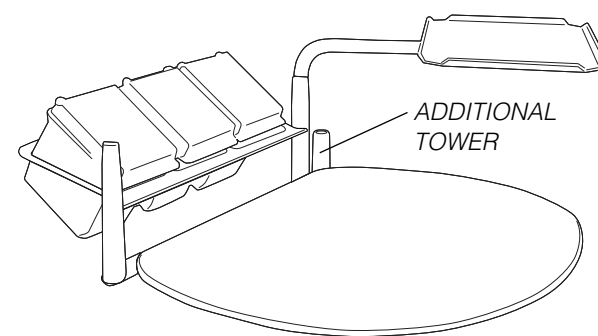


**7000 PRO SERIES DELIVERY**

**BIN PLATE INSTALLATION (FIG. 1).**

**STEP 1:** Install brackets under the chassis.

**STEP 2:** Attach bin plate to brackets.

**STEP 3:** Install bins into bin plate.

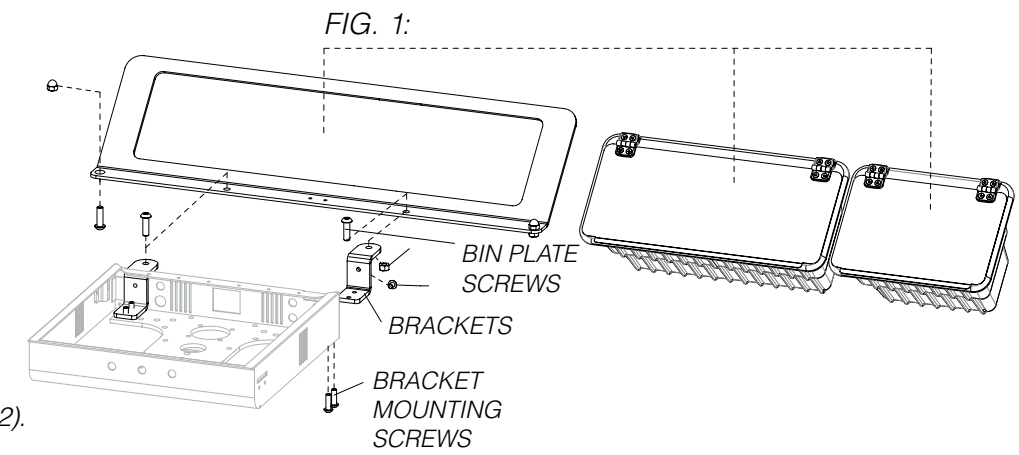


**PROCEDURE SHELF INSTALLATION (FIG. 2).**

**STEP 1:** Attach shelf posts to bin plate.

**STEP 2:** Thread upper towers into shelf posts.

**STEP 3:** Insert plugs into towers.

FIG. 2:

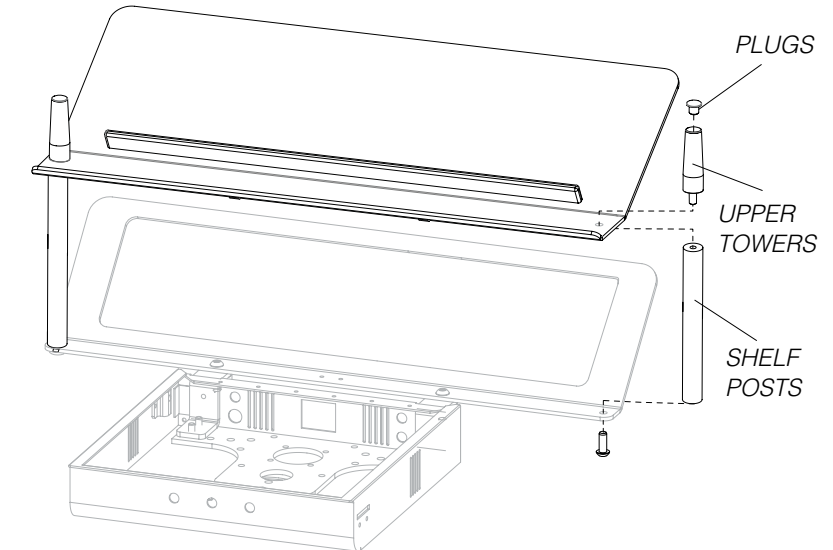


**SWIVEL TRAY INSTALLATION (FIG. 3).**

**STEP 1:** Install link to bottom of procedure shelf.

**STEP 2:** Attach tower to link and insert tray arm.

FIG. 3:

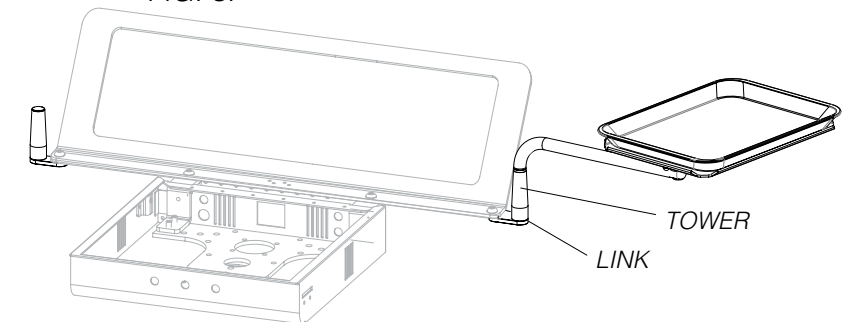


**MONITOR MOUNT INSTALLATION (PROCEDURE SHELF REQUIRED) (FIG. 4).**

**STEP 1:** Remove tower plug.

**STEP 2:** Insert monitor mount.

FIG. 4:

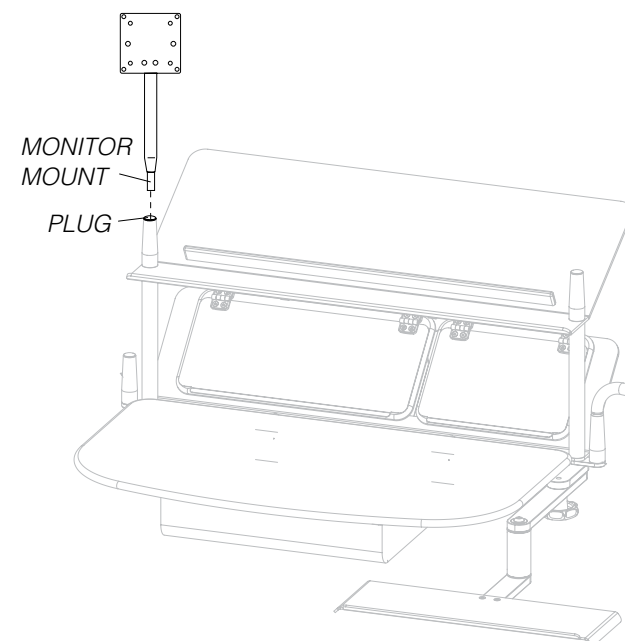

**KEYBOARD TRAY INSTALLATION (FIG. 5).**

**STEP 1:** Attach L-brackets under bin plate.

**STEP 2:** Using knob attach keyboard holder.

FIG. 5:

