

StarDental® Instruments

Junk-Out™ High-Speed Chuck Cleaner Instructions for Use

Clean chuck once a week

Shake well before use

Install brush to handle. See diagram at right.

Use one new brush per cleaning application. **Do not reuse brushes.**

Remove bur from chuck

Dip brush into bottle. Removing excess cleaner from the brush

Insert brush into bur end of turbine


Clean gently by rotating and inserting the brush in and out of the turbine.

Repeat process if necessary.

Reseal cap on bottle immediately after use to reduce evaporation.

Kit Reorder Part Number: StarDental® Part Number 263790.

Replacement brushes are available: StarDental® Part Number 264970.

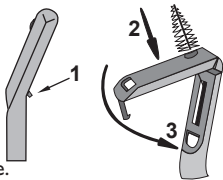

StarDental®
1816 Colonial Village Lane
Lancaster, PA 17601 U.S.A.
Made & Printed in the USA
©2018—DentalEZ Inc.

