

# Forest DENTALEZ

All controls are located on the front panel.

## AUTOMATIC HANDPIECE CONTROL

### MASTER ON/OFF (FIG. 1)

Turns on the air and water for the entire system. This toggle controls the air-actuated master valves in the utility center. Flip the toggle right (on) to activate the system. Always flip the toggle left (off) whenever the unit is not in use.

### HANDPIECE PRESSURE GAUGE (LOCATED INSIDE CONTROL HEAD)

Indicates drive air pressure to the handpiece.

### DRIVE AIR PRESSURE ADJUSTMENT (FIG. 2)

Allows handpiece to be adjusted to its manufacturer's recommended operating pressure. Turn the adjusting screw counter clockwise to increase the pressure, clockwise to decrease the pressure.

### WATER COOLANT FLOW ADJUSTMENT (FIG. 3)

Allows individual adjustment of water coolant to be made to the handpieces. Turn the knob counter clockwise to increase the flow, clockwise to decrease the flow.

### HANDPIECE FLUSH (FIG. 4)

Flushes the handpiece dental unit water lines (DUWL).

**⚠ WARNING:** Refer to Forest™ Dental Unit Water Line Maintenance Operator's Guide for detailed procedures on maintaining DUWL.

### AIR COOLANT FLOW CONTROL (FIG. 5)

Allows adjustment to be made to the handpieces. Turn the knob counter clockwise to increase the flow, clockwise to decrease the flow.

### WATER COOLANT ON/OFF (LOCATED ON THE FOOT CONTROL)

Activates handpiece water coolant system. Flip the toggle toward the red dot to enable the water coolant.

### PNEUMATIC BRAKE (FIG. 6)

Activates the flexible arm brake. Button is located on the handle on the control head. Press the button to release the brake.

### LID REMOVAL (FIG. 7)

With the lid directly in front of you:

**STEP 1:** Spread left and right side of lid outwards and up.

**STEP 2:** Reposition the whip arms forward with left arm.

**STEP 3:** Grab hold of the back (center) of lid and pull up with right hand.

**!** NOTE: Tool kit located inside control head for external adjustments.

