

Forest DENTALEZ

ALERT SYMBOL KEY

The safety notices associated with the following should be given special attention when they appear in maintenance, operating and emergency procedures in the guide.

⚠ WARNING indicates that the personal safety of the patient, end user or technician could be compromised by disregarding the WARNING. Not following instructions may result in an injury.

⚠ CAUTION indicates that a particular procedure or precaution must be followed to avoid system irregularity or possible damage to the product.

! NOTE indicates special information to improve the ease of maintaining the product, or to clarify important information.

POWER


Turning ON/OFF massaging unit.
Unit may also be turned ON by pressing any ZONE 1-3 key.

ZONES

(Green light indicates active Zone)
1 - Lower Back
2 - Upper Left Back
3 - Upper Right Back

INFLATE | DEFLATE

To activate press SELECT then press and hold INFLATE or DEFLATE to desired INFLATION/DEFLATION level.

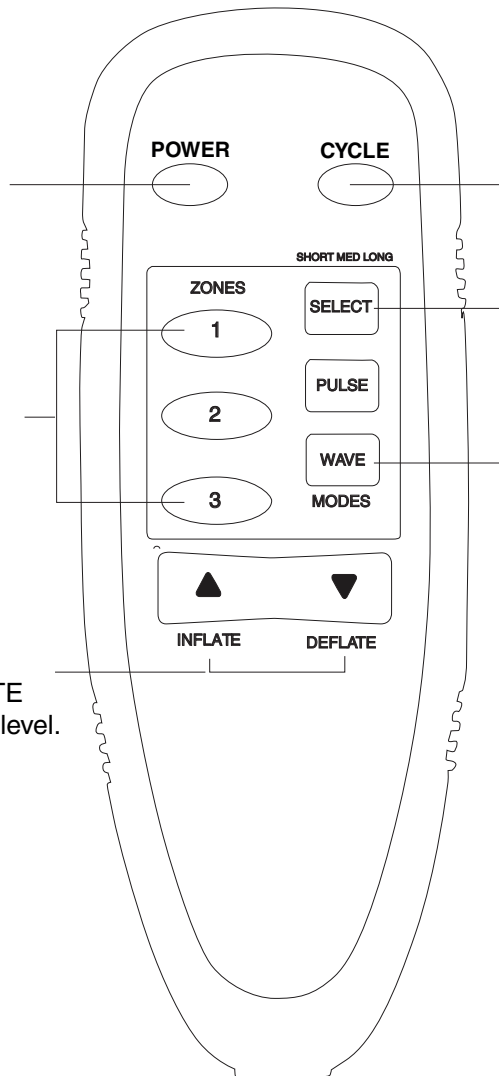

CYCLE

Works only when PULSE or WAVE is activated. Press to activate a SHORT, MEDIUM, or LONG cycle.

MODE

(Green light indicates active Mode)

ZONE

Press SELECT then any zone 1-3 to activate. Press any zone 1-3 to cancel.

PULSE

Massage simulating a pulsing action. Press PULSE followed by any zone 1-3. Press any zone 1-3 to cancel.

WAVE

Massage simulating a wave-like action. Press WAVE followed by any zones 1-3. Press any zone 1-3 to cancel.

! NOTE: System will deflate when unit is turned OFF.