

# Forest DENTALEZ

**ASSISTANT'S (FIG. 1)**

All controls are located underneath the work surface.

**DUO DELIVERY (REFER TO CONTROL HEAD OPERATION)**

All controls are located on the control head.

**OPTIONAL QUICK DISCONNECT (FIG. 2)**

Connection for additional water and/or air source.

**AIR FLOW ADJUSTMENT FOR QUICK DISCONNECT (FIG. 3)**

Allows adjustment of air through the quick disconnect. Turn knob counter clockwise to increase flow, clockwise to decrease flow.

**WATER FLOW ADJUSTMENT FOR QUICK DISCONNECT (FIG. 4)**

Allows adjustment of water through the quick disconnect. Turn knob counter clockwise to increase flow, clockwise to decrease flow.

**LEVELING ADJUSTMENT (WALL MOUNT) (FIG. 5)**

Adjust (4) leveling set screws to level.

**LEVELING ADJUSTMENT (BENEATH COUNTER MOUNT) (FIG. 6)**

Adjust (4) leveling set screws to level.

**DRIFT SET SCREWS (FIG. 6)**

Loosen or tighten (3) drift set screws.

**HEIGHT ADJUSTMENT (LOWERING) (FIG. 7)**

- STEP 1: Remove umbilical and retainers.
- STEP 2: Lift work surface off arm.
- STEP 3: Loosen and remove washer and nut.
- STEP 4: Flip arm assembly (see illustration below).
- STEP 5: Reinstall arm (by tightening the washer and nut).
- STEP 6: Reattach work surface back on arm.
- STEP 7: Route umbilical and reinstall retainers.

FIGURE 7: HEIGHT ADJUSTMENT

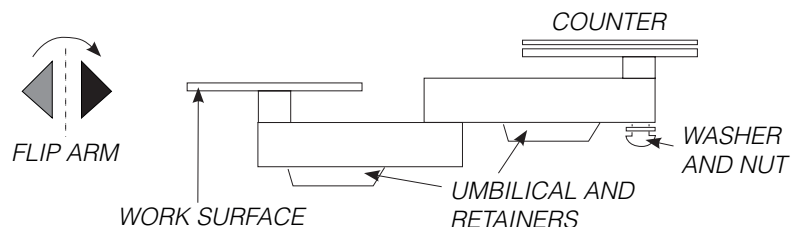
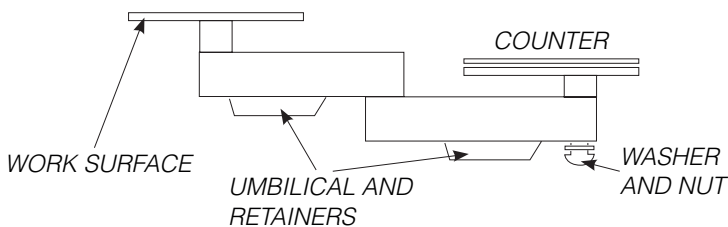


FIG. 1: CONTROLS UNDERNEATH THE WORK SURFACE

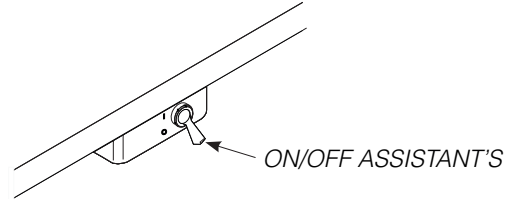


FIG. 2: QD CONTROLS UNDERNEATH THE WORK SURFACE (OPTION NOT STANDARD)

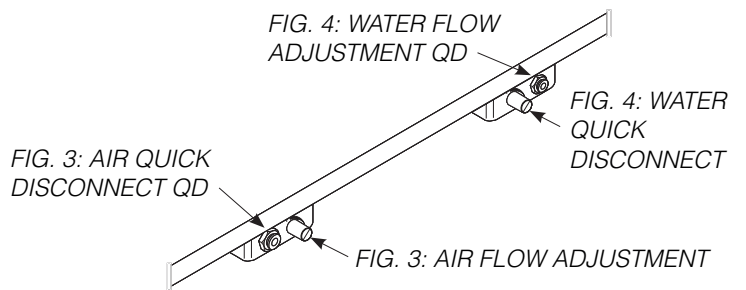


FIG. 5: WALL MOUNT

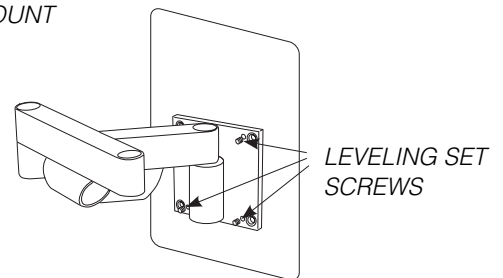


FIG. 6: BENEATH COUNTER

