

# Forest

## DENTALEZ



Refer to Forest™ General Information Operator's Guide for Glossary of Symbols, Terms, EMC/Electrical Safety Declaration and User Guidance.

**INDICATIONS FOR USE/INTENDED USE** The dental stool is intended to position the dental practitioner for access and visibility to the oral cavity during diagnostic and therapeutic treatment in a clinical environment. There are no contraindications for this product.

**⚠ WARNING:** Refer to Forest Operatory Equipment Asepsis Operator's Guide for surface and barrier protection procedures.

**⚠ CAUTION:** Federal law restricts this device to sale by or on the order of a dental practitioner licensed by the law of the State in which he/she practices to use or order the use of the device.



### 6176 DOCTOR'S STOOL AND 6187 ASSISTANT'S STOOL (FIG. 1)

**STEP 1:** Lift up on front right lever to adjust seat height (FIG. 1.1).

**STEP 2:** Lift up on rear right lever to adjust seat tilt (FIG. 1.2).

**STEP 3:** Rotate backrest height adjustment knob counter clockwise to loosen backrest. Once desired height is reached, rotate backrest height adjustment knob clockwise to lock backrest in place (FIG. 1.3).

**STEP 4:** Lift up on left lever to adjust backrest tilt. Push down on left lever until lever locks into place to actuate the free floating backrest (FIG. 1.4).

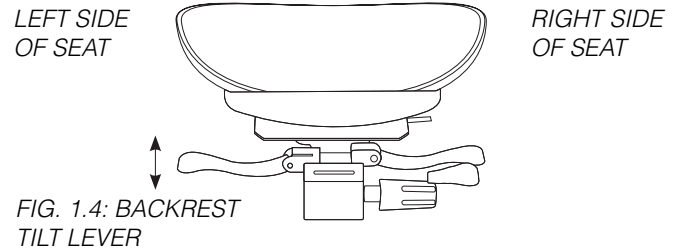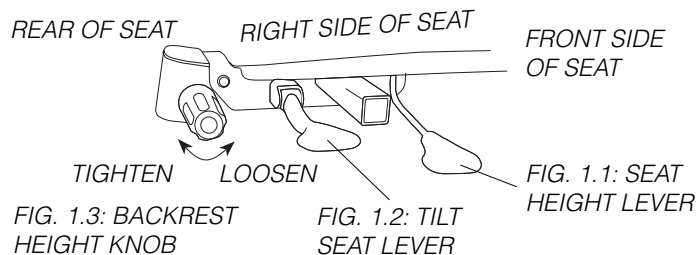

### 6187 ASSISTANT'S STOOL (FIG. 2)

**STEP 1:** Adjust torso support arm height by turning adjustment knob counter clockwise to loosen and clockwise to retighten (FIG. 2.1).

**STEP 2:** Toggle ratcheting arm lever in the direction you want the arm to move (FIG. 2.2).

