

# Forest DENTALEZ



Refer to Forest™ General Information Operator's Guide for Glossary of Symbols, Terms, EMC/Electrical Safety Declaration and User Guidance.

**INDICATIONS FOR USE/INTENDED USE** The dental stool is intended to position the dental practitioner for access and visibility to the oral cavity during diagnostic and therapeutic treatment in a clinical environment. There are no contraindications for this product.


**WARNING:** Refer to Forest Operatory Equipment Asepsis Operator's Guide for surface and barrier protection procedures.

**CAUTION:** Federal law restricts this device to sale by or on the order of a dental practitioner licensed by the law of the State in which he/she practices to use or order the use of the device.

**6156 DOCTOR'S ECONOMY STOOL, 6157 ASSISTANT'S ECONOMY STOOL AND 6160 ASSISTANT'S DELUXE STOOL (FIG. 1)**

**STEP 1:** Rotate backrest height adjustment knob counter clockwise to loosen backrest. Once desired position is reached, rotate backrest height adjustment knob clockwise to lock backrest in place (FIG. 1.1).

**STEP 2:** Lift up on right lever to adjust seat height. Release lever to lock in place (FIG. 1.2).

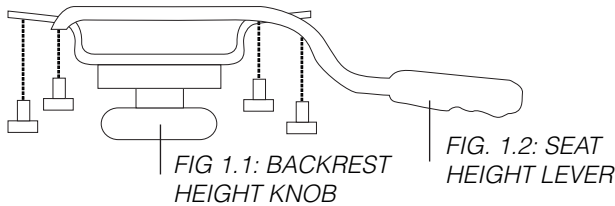



**6159 DOCTOR'S DELUXE STOOL AND 6161 ASSISTANT'S DELUXE STOOL (FIG. 2)**

**STEP 1:** Lift up on front right lever to adjust seat height (FIG. 2.1).

**STEP 2:** Lift up on rear right lever to adjust seat tilt (FIG. 2.2).

**STEP 3:** Rotate backrest height adjustment knob counter clockwise to loosen backrest. Once desired height is reached, rotate backrest height adjustment knob clockwise to lock backrest in place (FIG. 2.3).


**STEP 4:** Lift up on left lever to adjust backrest tilt. Push down on left lever until lever locks into place to actuate the free floating backrest (FIG. 2.4).

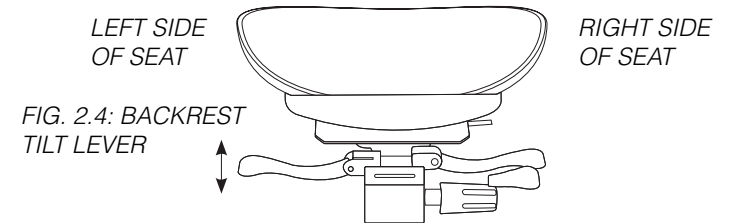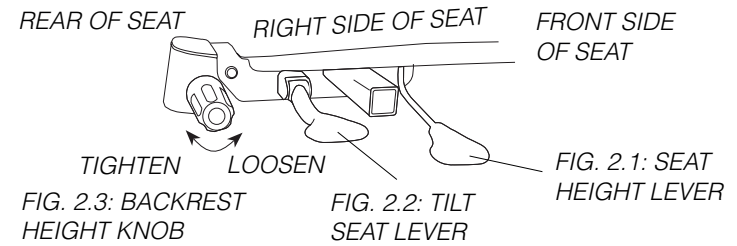

**6160 AND 6161 ASSISTANT'S DELUXE STOOL (FIG.3)**

**STEP 1:** Adjust torso support arm height by turning adjustment knob counter clockwise to loosen and clockwise to retighten (FIG. 3.1).

**STEP 2:** Toggle ratcheting arm lever in the direction you want the arm to move (FIG. 3.2).

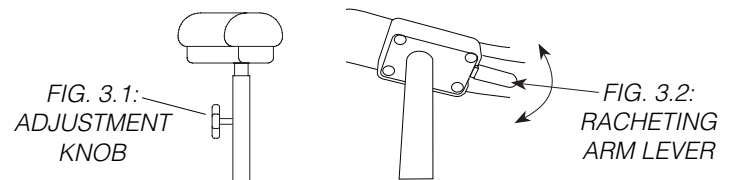

**DTE OREGON, INC.** DENTALEZ.COM

6200 NE Cherry Drive, Hillsboro, OR 97124 USA (TF) 800 . 423 . 3555 (T) 503 . 640 . 3012 (F) 503 . 693 . 9715

