



Sparkl[®]

*** DENTAL
INSTRUMENT
CLEANER**

USES: CLEANS ALL MAKES AND TYPES OF:

- Diamond Instruments
- Perforated and Hollow Instruments
- Carbide and Steel Burs
- Root Canal Files, Reamers & Broaches
- Hand Instruments: Explorers, Scalers, Currettes, Excavators, & Pluggers

DANGER:

KEEP OUT OF REACH OF CHILDREN.

See first aid statement and additional precaution in the Safety Data Sheet included with this product.

One Liter (33.8oz = One Quart, 1.8 Ounces)
Concentrate: Makes Ten Quarts of Solution
Item No. 205472

StarDental[®] Instruments

USES: Cleans all makes and types of:

- diamond instruments
- perforated and hollow instruments
- carbide and steel burs
- root canal files, reamers & broaches
- hand instruments: explorers, scalers, currettes, excavators & pluggers

Will not affect bond of diamond instruments, dull carbide bur cutting edges, corrode or rust metals or deteriorate plastics. (Do not use with mounted carborundum stones or unprotected aluminum.)

DIRECTIONS:

To remove all adhering blood and serous exudate before sterilization or disinfection, wash instruments in a dilution of one part **Sparkl[®]** Concentrate to nine parts water.

Use in **StarDental[®]** Root Canal Organizer, Item #205754, to clean and store files and reamers. Dilute **Sparkl[®]** Concentrate to the formula outlined above. Keep organizer sponge thoroughly saturated with this solution. To slow evaporation, mix 2/3 of solution with 1/3 propylene glycol.

After removing instruments from **Sparkl[®]** solution, thoroughly rinse instruments in water before autoclaving, disinfecting or using.

When bottle is empty, rinse thoroughly with water and discard in accordance with all State, Federal and Local Regulations.

Keep from Freezing: If exposed to low temperatures, a precipitate may form in the bottle. To dissolve precipitate, warm bottle in hot water and shake vigorously.

PRECAUTION: Do not expose to temperatures above 54° C (130° F).

⚠ DANGER: KEEP OUT OF REACH OF CHILDREN. See first aid statement and additional precaution in the Safety Data Sheet included with this product.

DISSOLVES DENTAL DEBRIS, BLOOD AND SEROUS EXUDATE:

Ordinary tooth debris, blood and serous exudate are dissolved and removed within one hour. Stubborn impacted debris may require longer immersion. Keep instruments immersed until clean. Items may be stored overnight without damage. Cleaning action is chemical. Usually scrubbing is not required. Change solution every day or two or when it becomes discolored. **Sparkl[®]** is odorless, non-staining, rust-preventing and non-volatile.

NOT FOR USE IN ULTRASONIC BATH CLEANERS.

⚠ WARNING:

Corrosive to human tissue. Causes eye damage and skin irritation. Protect eyes and skin when handling. Harmful if swallowed. Avoid contamination of food.


Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

FIRST AID:

Immediately flush eyes or skin with plenty of water. Continue procedure for at least 15 minutes. Call a physician in the case of eye contact.

If clothing is contaminated, remove and wash before reuse.

If swallowed, promptly drink a large quantity of milk, egg white, gelatin solution or, if these are not available, drink a large quantity of water. Avoid alcohol. Call a physician immediately.


StarDental[®]
1816 Colonial Village Lane
Lancaster, PA 17601 U.S.A.
Printed in the U.S.A.
©2018 DentalEZ, Inc.

097058 Rev Q Date of Issue 2018-03-08

866.DTE.INFO (383.4636)
www.DentalEZ.com